Ultrasonic & Powder Combination System

Varios Combi Pro

OPERATION MANUAL

Please read this OPERATION MANUAL carefully before use, and file for future reference.

Thank you for purchasing Varios Combi Pro. Please read this Operation Manual carefully before use for operating instructions and care and maintenance guidelines so that you can use it for many years to come through a correct use. Keep this Operation Manual within easy reach of users for future reference.

– Contents –

1. User and Intended Use 1

2. Precautions for handling and operation 1

3. Accessory List 5

4. Names of Components 6

5. Installation 8

6. Check before treatment 11

7. Using the Ultrasonic Mode 12

8. Using the Powder Mode 16

9. Maintenance 20

10. Care and Maintenance 25

11. Other Functions 30

12. Periodical Maintenance Checks. 31

13. Error Code. 31

14. Troubleshooting 32

15. Specifications 35

16. Classification of Equipment. 35

17. Operation Principle. 36

18. Symbol 36

19. Warranty 36

20. Option Parts List 36

21. Spare Parts List 37

22. Disposing product 38

23. EMC Information (Electromagnetic Compatibility Information) 38

* Some sections are indicated with the mark(s) below to show the mode(s) related to the section.

-  Description regarding Ultrasonic Mode
-  Description regarding Prophy Mode under Powder Mode
-  Description regarding Perio Mode under Powder Mode

1 User and Intended Use

User: Dentist, dental hygienist

Intended Use: Tooth surface cleaning, removal of plaque or calculus, tooth resection, expansion or cleaning of root canal, removal of foreign substances, root canal obturation, enhancing adhesion of fixed restorations, incision or removal of periodontal tissue

2 Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

⚠ WARNING

<Ultrasonic Mode and Powder Mode>

- Do not handle the power cord with wet hands. Wet hand contact with electricity may result in an electric shock.
- Keep away from explosive substances and flammable materials.
- If the product overheats or smells of burning, immediately turn off the power and disconnect the main power cord. Contact your Authorized NSK Dealer.
- TO PREVENT ELECTRIC SHOCK, use a main electrical outlet that is earthed.
- Do not operate close to patients with cardiac pacemakers as there is a danger that it may affect the pacemaker.
- The system functions normally in the environment where the temperature is at 0 - 40°C (32 - 104°F), humidity at 35 - 75% RH, and no moisture condensation in the Control Unit. Using in conditions outside these limits may cause malfunction.
- Be careful not to get water or liquid disinfectant in the Control Unit. This could cause short circuits and lead to fire and/or electric shock.
- Repeatedly turning the Main Power Switch ON and OFF may blow a fuse.
- If water droplets, chemicals, solvents or antiseptic solution come into contact with the Control Unit, Handpiece, Handpiece Cord, Handpiece Hose, etc., wipe them off immediately. Failure to do so may result in discoloration or deformation.
- Connect to a public water system that provides drinkable water at a water pressure between 0.1 and 0.3MPa. Using with an incorrect water pressure may result in insufficient water supply, inoperability, or malfunction.
- Use a water softener, etc. to lower the hardness of the water, because using water with a high degree of hardness may cause mineral or limescale deposits in the water circuit, which can lead to problems such as insufficient water levels, operation problems, or breakdown. When doing so, use a Water Tube with a Water Filter to prevent device breakdown due to the foreign material in tap water, and refer to "10-3 Changing the Water Filter" to replace the Water Filter as required.
- Do not sterilize using ultraviolet light. Doing so may result in discoloration of the Handpiece.
- Hold the plug when pulling out the Handpiece Cord or Handpiece Hose. Pulling the Cord or Hose may cause disconnection or damage.

<Ultrasonic Mode>

- Always use the water supply. Insufficient water supply results in overheating, which may damage the surface of the patient's tooth.
- Do not allow the patient or assistant to look directly into the LED of the Handpiece or Handpiece Cord. Doing so may damage eyesight.

<Powder Mode>

- Do not use on the following patients.
 - Those with medical complications or allergies
 - Those who have pre existing conditions (Eg Cardiac, Pulmonary, Renal disturbance or High blood pressure)
 - Those with respiratory diseases.
 - Those with pathologically deep periodontal pockets (6mm or more) or mucosal lesions. Prolonged application of the spray to soft tissue or salivary glands may cause powder air to enter soft tissue or salivary glands, resulting in emphysema.
 - Those with severe digestive ulcers.
 - Those with renal dysfunction.
 - Those with lung or cardiac disturbance or ventilatory impairment.
 - Those with an artificial heart valve or artificial joint
 - Those with allergies. Depending on the patient's constitution, allergy symptoms or stomatitis may be caused in very rare cases. If such symptoms are detected, stop using the product immediately and seek medical attention.
- During use, both the operator and the assistant should always wear protective glasses and protective face masks. Also, continuously vacuum and collect ejected powder during use. If any powder gets into the eyes, eyes wash out immediately with copious amount of water and consult an ophthalmologist.
- Set the supply air pressure to 0.55 - 0.75MPa and use clean, dry air. Using with the incorrect pressure may result in insufficient spray pressure (power), inoperability or malfunction.

<Prophy Mode>

- Do not spray directly onto the cement in the root canal, decalcified enamel, filling, margin of a prosthesis or filling.

⚠ WARNING

<Perio Mode>

- When using Perio Mode, pay attention not to cause emphysema as air containing powder is sprayed into periodontal pockets. If emphysema is caused by treatment using Perio Mode, stop using this product immediately and seek medical attention.
- Do not use on the following patients. Using on periodontal pockets for these patients may result in bacteremia.
 - Patients with endocardial inflammation
 - Patients who have a contagious illness or disease
 - Patients with an immune deficiency disease (neutropenia, granulocytopenia, agranulocytosis, diabetes, hemophilia)
 - Patients undergoing irradiation treatment, chemotherapy, or antibiotic treatment
- Do not use on the following patients. (Excluding the using without the Nozzle Tip)
 - Those with damage or abnormality in the oral cavity.
 - Those with hyperemia, hemorrhage or inflammation in the oral cavity.
 - Those that tend to develop inflammation or sores in the oral mucous membrane.
- Use this product only for maintenance after basic periodontal treatment. Never use it for other purposes.
- Use Perio Mode only on periodontal pockets that meet the following conditions:
 - Periodontal pockets caused by gum disease
 - Periodontal pockets of less than 6mm that have undergone basic periodontal treatment
- Check the depth of periodontal pockets with a measurement probe or photography before use.
- Use the designated cleaning powder only. Use of other powders may result in malfunction, substandard performance or damage to the substantia ossea and dentine, or may cause emphysema. NSK cannot accept any liability for issues that occur as a result of the use of powder other than those designated.
- Nozzle Tip for Perio Mode included in the product package is a sterile item. When using these products, follow the instructions below.
 - Nozzle Tip is a single-use item. Do not reuse nor resterilize it as product breakage or infection could occur.
 - Check before use that the package is NOT open nor damaged. If the package is open or damaged, do not use that Nozzle Tip as it may be damaged or contaminated.
 - Observe the use-by date written on the Nozzle Tip package label. Do not use expired products as product breakage or infection could occur.

⚠ CAUTION

<Ultrasonic Mode and Powder Mode>

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- The end user shall be responsible for any judgment that relates to the application of this product to a patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- This device is for indoor use only.
- Keep the Control Unit on a level surface.
- Do not attempt to disassemble the product nor tamper with the mechanism except as recommended by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product. The Glass Rod and Handpiece may be damaged if the Ultrasonic Handpiece is dropped.
- Operators and all others in the area must wear eye protection and a mask while using this product.
- Should the product function abnormally, cease operation immediately and contact your Authorized NSK Dealer.
- Push the Handpiece securely and fully onto the Handpiece Cord or Handpiece Hose when connecting.
- Do not autoclave the Handpiece Cord or the Handpiece Hose. Use alcohol to disinfect the Cord or Hose after use.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the product.
- This product is not delivered in a sterile condition, except for the Nozzle Tip (option) and must be autoclaved prior to use. Refer to “9-3 Sterilization” for details.
- Perform regular function and maintenance checks.
- If the product is not used for a long period check it is functioning correctly before using on a patient.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during surgery.
- Always replace the fuse with the same specified rating.
- This product is rated Medical Electrical equipment. EMC (Electromagnetic compatibility) is described in the documentation included.
- Installation and use of this product requires special precautions regarding EMC according to the EMC information.
- Portable and mobile RF communications equipment can affect Medical Electrical equipment. Do not use RF equipment near the product.
- The use of ACCESSORIES such as handpieces and cables, other than those specified by the manufacturer, with the exception of handpieces and cables sold by the manufacturer of this product as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of this product.
- This product should not be used adjacent to, or stacked with, other equipment. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

⚠ CAUTION

- The system may present a possibility of malfunction when used in the presence of an electromagnetic interference wave. Do not install the system in the vicinity of any device which emits magnetic waves. Turn off the Main Power Switch of the system as an ultrasonic oscillation device or an electrode knife is located close to the vicinity of use.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.
- For safety's sake, install the Control Unit in a place where the AC Power Cord can be easily removed. (It is possible to disconnect the control unit from the power source by removing the AC Power Cord.)

<Ultrasonic Mode>

- The temperature of the Tip surface may reach 65°C without using coolant water. To avoid this event, be sure to use coolant water.

<Powder Mode>

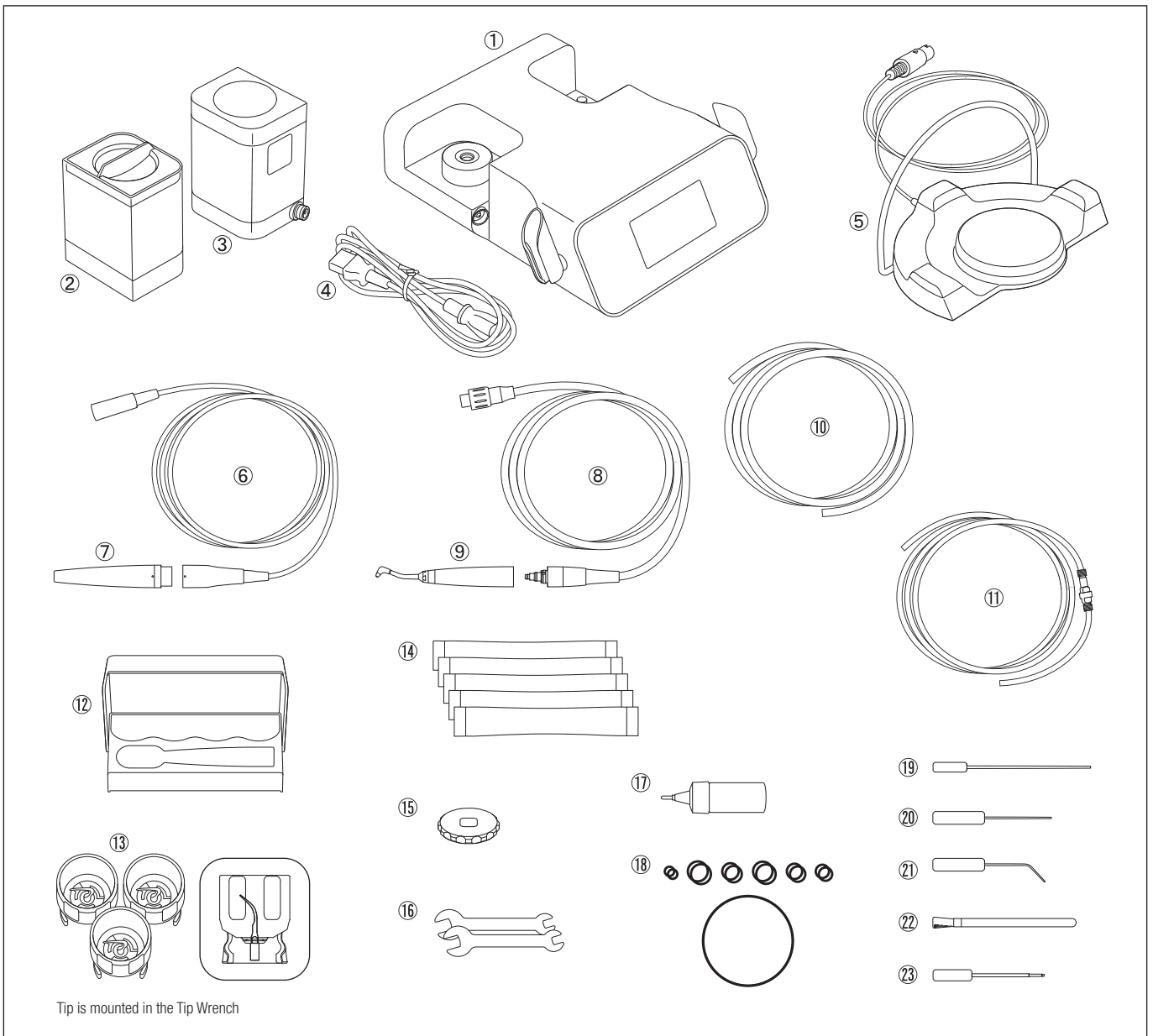
- Be very careful to ensure cleaning powder does not enter the patient's mucosal areas (eyes, nose, etc.) other than the oral cavity. Also, protect the face with a towel or protective glasses, etc., to prevent the cleaning powder from getting into the patient's eyes.
* If the patient wears contact lenses, have the patient wear safety glasses or remove his/her contact lenses before treatment.
- Make sure the compressed air supply is clean and dry. Water or oil content mixed in the air supply may cause solidification of the cleaning powder inside the product.
- Securely attach the Chamber Cap to prevent the chamber cap from being blown off, which may lead to an injury.
- Do not open the Chamber Cap during use. Doing so may result in powder being ejected from the chamber cap due to high pressure in the chamber.
- Before connecting the Handpiece Hose, Chamber and Handpiece, or the Chamber and Control Unit, be sure to blow off all water from the connections of the Hose and the Chamber with dry air. Failure to do so may result in moisture entering the air supply, causing solidification of the cleaning powder inside the product.
- Depending on the use environment, the temperature of the Powder Handpiece surface may reach 50°C.
- If too much cleaning powder is put into the Chamber, the proper amount of powder may not be sprayed at the start of use.
- Powder scatters easily because it is very fine. Fill the Chamber slowly and be sure to prevent the powder from scattering.
- Powder stored for a long time in opened packaging may become moist and clog. Always use new, dry powder. When using bottled powder (option), securely fasten the bottle lid after use.
- Apply Vaseline, etc., to the patient's lips to prevent drying or chapping at the corners of the mouth.

NOTICE

- No special training is required for this device.
- Repeated autoclaving may cause the ultrasonic handpiece to become discolored due to heat. However, this is due to properties of the product and is not a problem in terms of quality.

3 Accessory List


No.	Part Name	Quantity	No.	Part Name	Quantity
1	Control Unit	1	14	FLASH pearl	5
2	Irrigation Bottle	1	15	Ring Wrench	1
3	Chamber (For Prophy use)	1	16	Spanner Wrench (5×8)	2
4	AC Power Cord	1	17	Turbine Oil	1
5	Foot Control	1	18	Spare O-Ring	7 types
6	Handpiece Cord	1	19	Cleaning Wire (L)	1
7	Ultrasonic Handpiece	1	20	Cleaning Wire (S)	1
8	Handpiece Hose (For Prophy use)	1	21	Cleaning File	1
9	Powder Handpiece (For Prophy use)	1	22	Cleaning Brush	1
10	Air Hose	1	23	Unlocking tool	1
11	Water Filter Set	1	24	Operation Manual (This manual)*	1
12	Sterilization case	1	25	Tip-Power Guide*	1
13	Tip and Tip Wrench	3	26	Tip Card*	1
			27	Varios Combi Pro Quick Operation Guide*	1

* Not shown in the illustrations above


NOTICE

- FLASH pearl and Perio-Mate powder (option) used in Powder Mode are referred to as “Powder” in this Operation Manual unless otherwise specified.
- Optional Perio Kit need to be purchased separately in order to use Perio Mode. Refer to “20 Option Parts List” for details of Perio Kit.

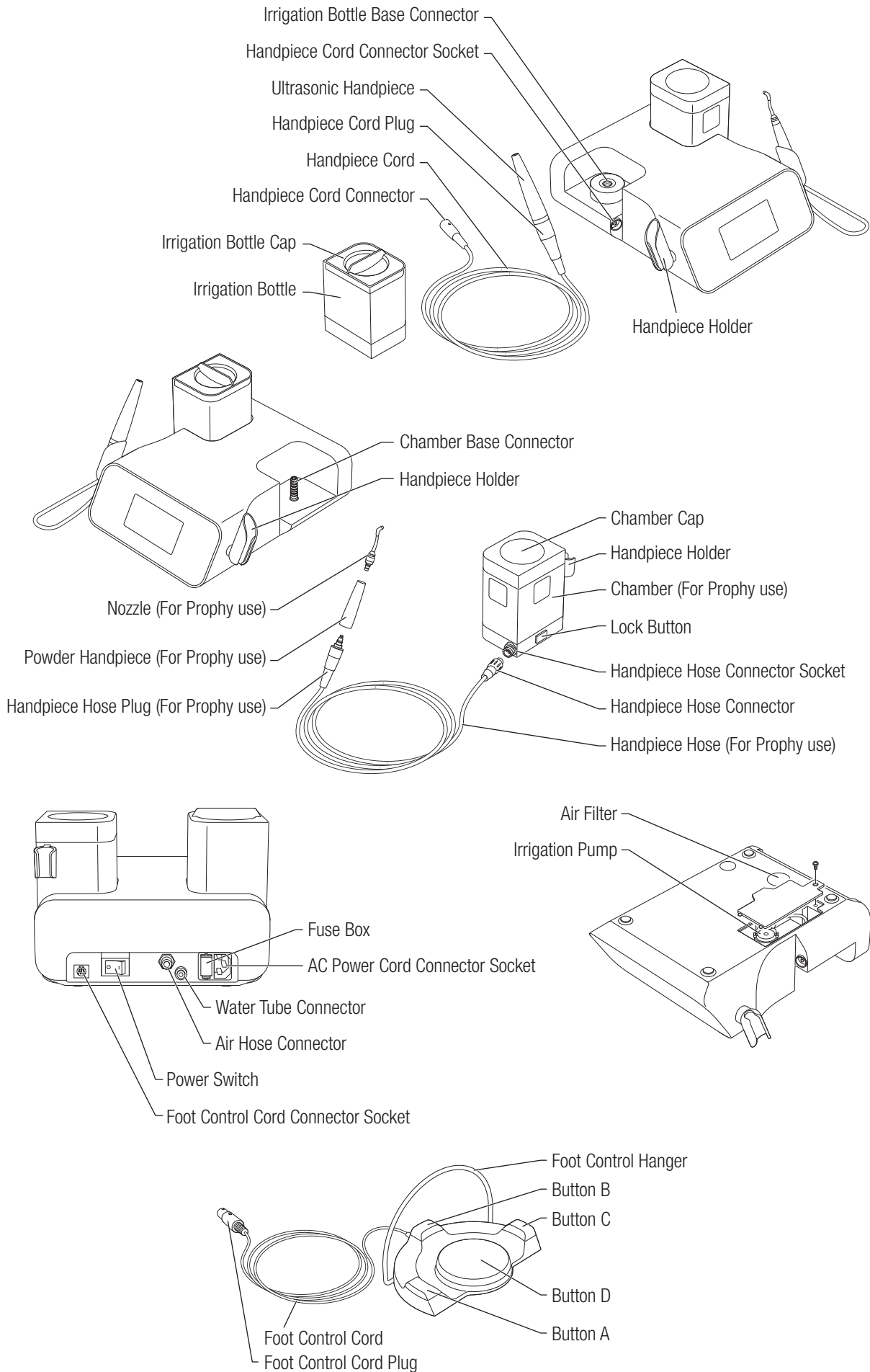4 Names of Components


4-1 Operation Panel

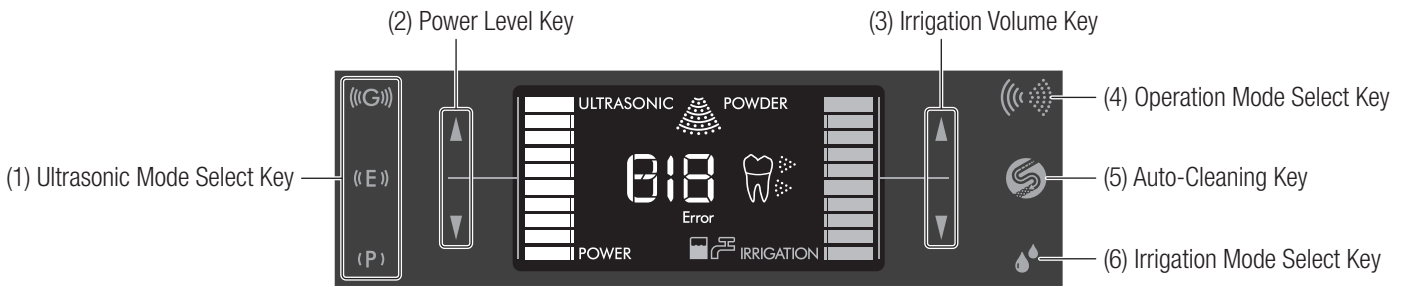
- | | |
|---|---|
| <p>(1) Ultrasonic Mode Select Key
Switches the Ultrasonic Mode between P (Perio), E (Endo), G (General). The strongest power level is G, followed by E and P. Power Level, Irrigation Level, and Irrigation Mode can be set for each mode.</p> <p>(2) Power Level Key
Sets the power level using the ▲/▼ keys.</p> <p>(3) Irrigation Volume Key
Sets the irrigation volume using the ▲/▼ keys.</p> <p>(4) Operation Mode Select Key
Switches between “Ultrasonic Mode” and “Powder Mode.”</p> | <p>(5) Auto-Cleaning Key
Starts Auto-Cleaning. Refer to “9-1-1 Auto-Cleaning” (Ultrasonic Handpiece) or “9-2-1 Auto-Cleaning” (Powder Handpiece) for details.
Ultrasonic Mode:
Sprays tap water for 15 seconds, then sprays water from the Irrigation Bottle for 30 seconds to clean the Ultrasonic Handpiece.
Powder Mode:
Sprays air and water at the same time for 10 seconds and then sprays only air for 35 seconds.</p> <p>(6) Irrigation Mode Select Key
Switches the Irrigation Mode (Bottle or tap water) for Ultrasonic Mode. Tap water is always used for Powder Mode.</p> |
|---|---|

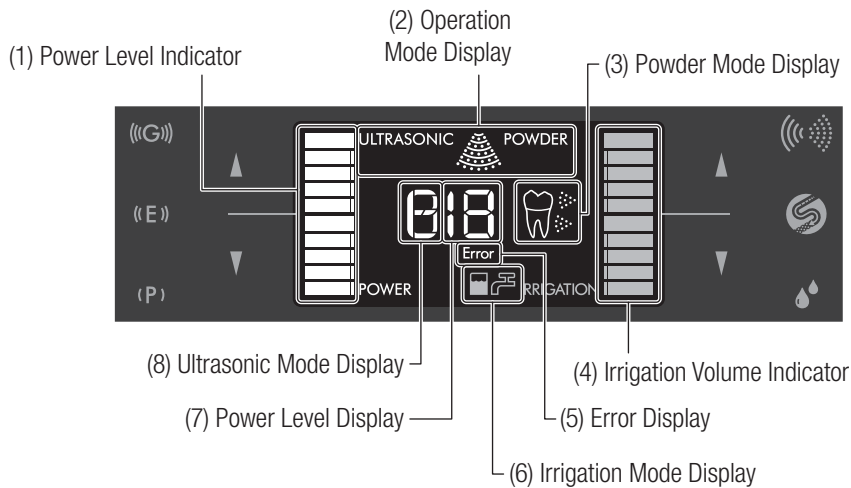4-2 LCD Panel

- | | |
|--|---|
| <p>(1) Power Level Indicator
Displays the current power on the Bar Graph Indicator.</p> <p>(2) Operation Mode Display
Displays the current Operation Mode (Ultrasonic or Powder).</p> <p>(3) Powder Mode Display
Displays the set chamber (Prophy or Perio) (only when Powder Mode is in use).</p> <p>(4) Irrigation Volume Indicator
Displays the selected Irrigation Level on the Bar Graph Indicator.</p> | <p>(5) Error Display
Displayed when an error occurs. Refer to “13 Error Code” for details.</p> <p>(6) Irrigation Mode Display
Displays the Irrigation Mode (Bottle or tap water) for Ultrasonic Mode. Tap water is always used for Powder Mode.</p> <p>(7) Power Level Display
Displays the current Power Level as a number.</p> <p>(8) Ultrasonic Mode Display
Displays the current Ultrasonic Mode (P/E/G).</p> |
|--|---|

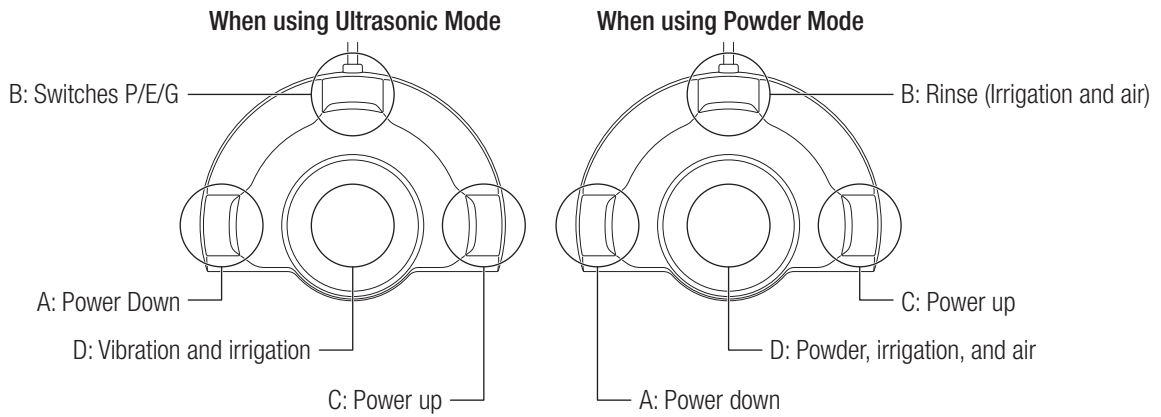4-3 Foot Control

Ultrasonic Mode

- A: Power Down
Reduces the power level.
- B: Switches P/E/G (Ultrasonic Mode)
Toggles Ultrasonic Mode in the order of P (Perio) → E (Endo) → G (General) → P (Perio)... every time the button is depressed.
- C: Power up
Increases the power level.
- D: Vibration and irrigation
Vibrates and irrigates while the button is depressed.

Powder Mode

- A: Power Down
Reduces the power level.
- B: Rinse (Irrigation and air)
Sprays water and air while the button is depressed.
- C: Power up
Increases the power level.
- D: Powder, irrigation, and air
Sprays powder, water, and air while the button is depressed.

5 Installation

5-1 Connecting the AC power cord

- 1) Turn the Power Switch to OFF before connecting the AC power cord.
- 2) Insert the AC Power Cord so that it correctly fits into the AC Power Cord Connector Socket at the back of the Control Unit (Fig. 1).
- 3) Plug the AC power cord into a wall outlet specified for medical use.

- CAUTION**
- Do not tug the AC power cord by holding and pulling its cord.
 - Make sure that the device has completely stopped before removing or inserting the AC power cord.

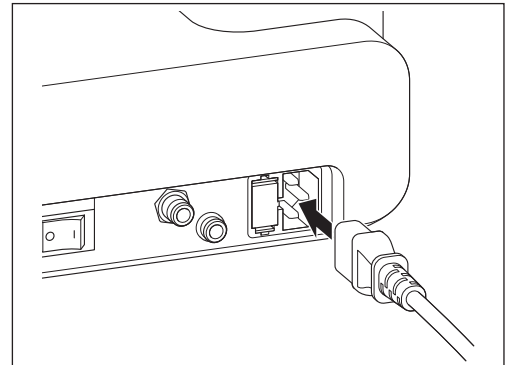
Fig. 1

5-2 Connecting the Air Hose

- 1) Insert one end of the Air Hose deep into the Air Hose Connector (Fig. 2).
- 2) Pull the Air Hose gently and check that the Hose does not come out.
- 3) To remove the Air Hose, pull the Air Hose out while pushing the blue ring toward the Control Unit (Fig. 3).

 • Use clean dry air. Powder may clog if the air contains moisture or oil.

CAUTION

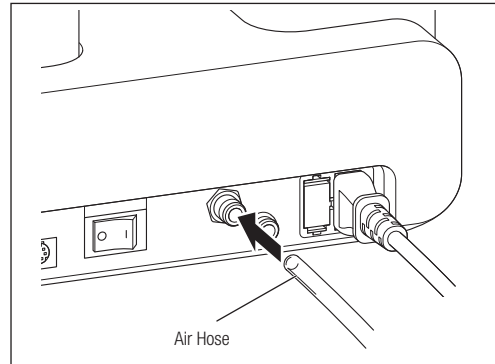
Fig. 2

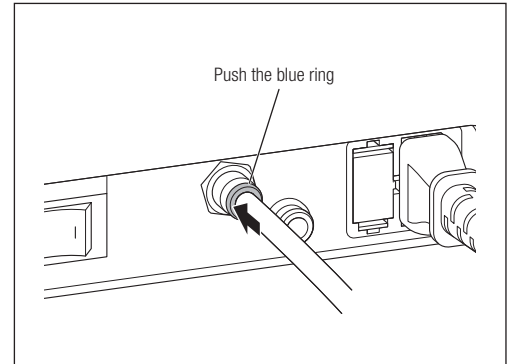
Fig. 3

5-3 Connecting the Foot Control

- 1) Insert the Foot Control Hanger so that it fits correctly into the holes at the rear of the Foot Control (Fig. 4).
- 2) Insert the Foot Control Plug into the Foot Control Cord Connector Socket with the ▲ mark facing upwards (Fig. 5).

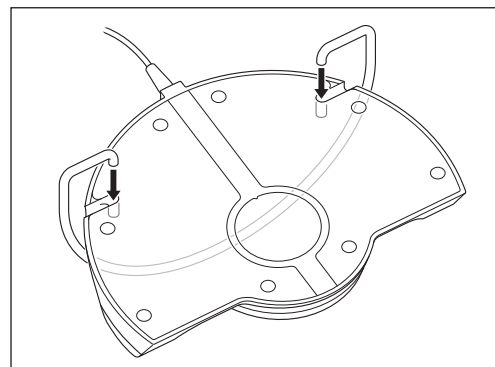
Fig. 4

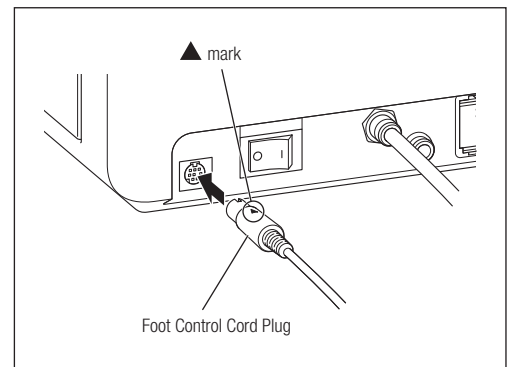
Fig. 5

5-4 Connecting the Handpiece Cord (Ultrasonic Handpiece)

Connect the Handpiece Cord Connector to the Handpiece Cord Connector Socket with the ▲ mark facing upwards (Fig. 6). Push it in all the way.

 • Check that the Handpiece Cord Plug is clean and dry before connection.

CAUTION

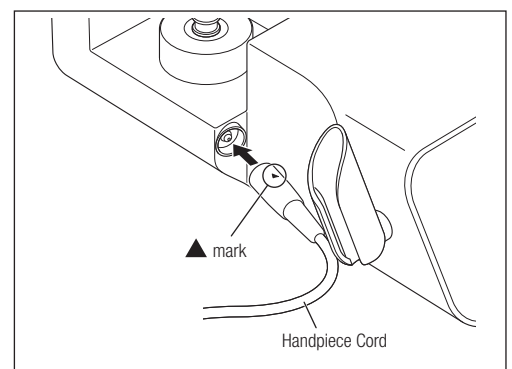
Fig. 6

5-5 Connecting the Handpiece Hose (Powder Handpiece)

- 1) Use air to dry the Handpiece Hose Connector Socket and Handpiece Hose Connector.
- 2) Insert the Handpiece Hose Connector so that it fits correctly into the hole on the Handpiece Hose Connector Socket and fasten the Connector nut (Fig. 7).

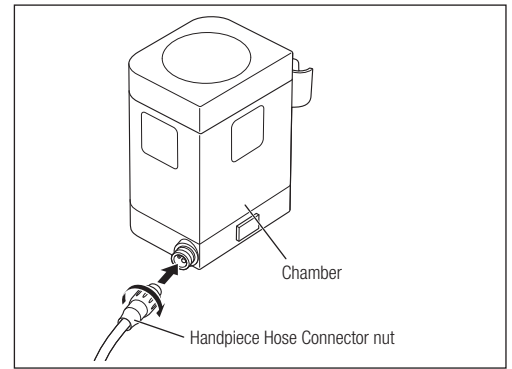
Fig. 7

5-6 Connecting and disconnecting the Handpiece

CAUTION • Be sure to only use the supplied Handpiece.

ULTRASONIC

5-6-1 Ultrasonic Handpiece

- (1) Connection
Align the dots on the Ultrasonic Handpiece and the Handpiece Cord Plug. Push the Handpiece straight into the connector (Fig. 8).
- (2) Disconnection
Hold the Ultrasonic Handpiece and the Handpiece Cord Plug and pull the Handpiece out.

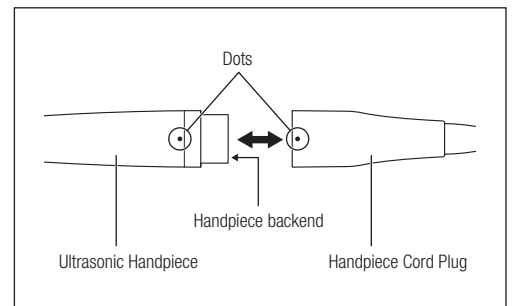
Fig. 8

WARNING • To avoid an electric shock, do not touch the electrical contacts of the handpiece backend and the handpiece cord.

CAUTION • Do not twist the Handpiece or Handpiece Cord when removing the Handpiece. Doing so may damage the Connector electrical contacts, resulting in malfunction.
• Before connecting the handpiece cord to the handpiece, face the handpiece connector side so light shines into it and check the end of the handpiece. If the light transmitted by the Optic Fiber End Face at the end is uniformly bright, it is functioning normally. If part or all is dark, the Optic Fiber End Face may be cracked or broken.

5-6-2 Powder Handpiece

- (1) Connection
 - 1) Use air to dry the connection between the Handpiece and Handpiece Hose Plug.
 - 2) Push the Powder Handpiece straight into the Handpiece Hose Plug (Fig. 9).
- (2) Disconnection
Hold the Powder Handpiece and the Handpiece Hose Plug, and pull the Handpiece out.

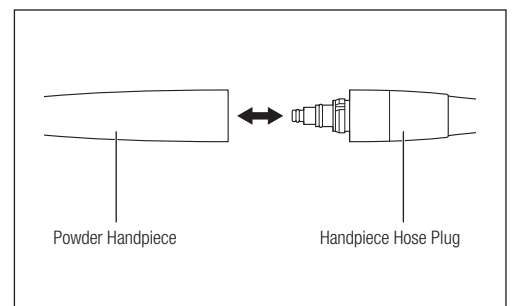
Fig. 9

5-7 Connecting the Water Tube

- 1) Insert one end of the Water Tube (the end with the water filter) deep into the Water Tube Connector (Fig. 10).
- 2) Connect the other end of the Water Tube to the water outlet on the dental unit.
- 3) Pull the Water Tube gently and check that the Tube does not come out.
- 4) To remove the Water Tube from the Water Tube Connector, pull the Water Tube out while pushing the green ring toward the Control Unit (Fig. 11).

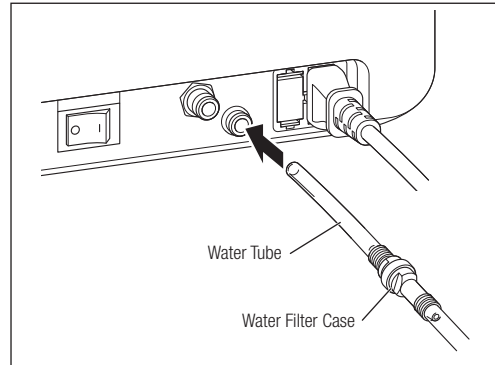
Fig. 10

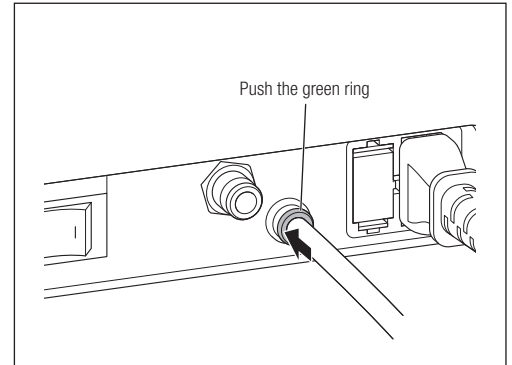
Fig. 11

WARNING

- Use a water softener, etc. to lower the hardness of the water, because using water with a high degree of hardness may cause mineral or limescale deposits in the water circuit, which can lead to problems such as insufficient water levels, operation problems, or breakdown. When doing so, use a Water Tube with a Water Filter to prevent device breakdown due to the foreign material in tap water, and refer to "10-3 Changing the Water Filter" to replace the Water Filter as required.

CAUTION

- If the water outlet on the dental unit has not been used for a long time, water may contain rust, etc. when it is run for the first time. Make sure that the water is clear before connecting the Water Tube.
- Insert the Water Tube firmly into the Water Tube Connector when connecting it to the Control Unit. Water may leak if the Tube is not inserted correctly.

6 Check before treatment

Before use, operate the product without a patient present to confirm there is no abnormal vibration, noise, heat, etc. Also check the following points.

- Ultrasonic Handpiece: LED lights and water flows.
- Powder Handpiece: Powder, air and water are supplied correctly.

If you notice anything unusual when checking or operating this product, stop using it immediately and contact your authorized NSK dealer. When you perform Auto-Cleaning, the water inside the Handpiece Cord/Hose is ejected, which may make stop water from flowing when using the Control Unit the next time. If that happens, set the Irrigation Volume to 10, and operate the Handpiece for about 30 seconds after water starts to come out. Refer to "14 Troubleshooting" if water does not start to come out after more than 20 seconds, even if you set the Irrigation Volume to 10.

7 Using the Ultrasonic Mode

7-1 Water System Setup

Using the Bottle

- 1) Remove the Dust Cover from the Irrigation Bottle Base Connector (Fig. 12).
- 2) Remove the Irrigation Bottle Cap and fill the Bottle water or disinfectant solution to the desired level.
- 3) Close the Irrigation Bottle Cap securely and insert the Bottle Joint straight into the Irrigation Bottle Base Connector (Fig. 13). Pull the Irrigation Bottle straight up to remove it.

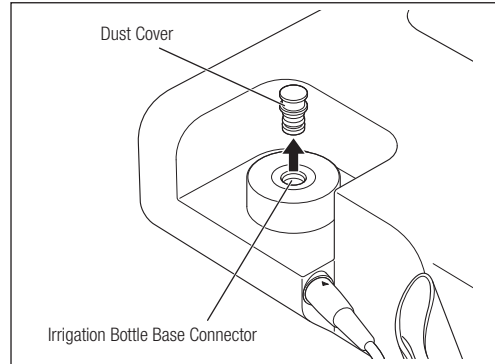
Fig. 12

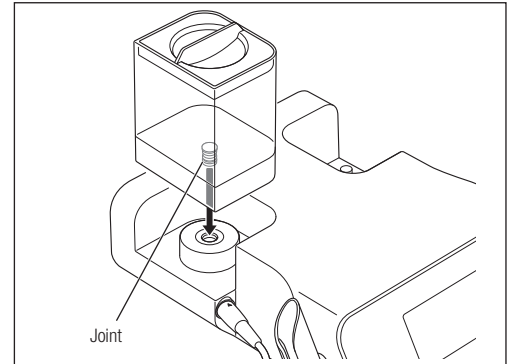
Fig. 13

CAUTION

- Only use the dedicated Varios Combi Pro series bottle.
- Insert the Irrigation Bottle straight. Failure to do so may damage the O-ring.
- Clean the Irrigation Bottle and the Irrigation Bottle Cap with clean water after use and before storage.
- Attach the Dust Cover to the Irrigation Bottle Base Connector when the Irrigation Bottle is not mounted.
- Ensure the liquid temperature is below 35°C.
- Do not put strong acids (such as highly acidic water, superacid water, or electrolysis water) or hypochlorous acid water in the Irrigation Bottle as they may cause water clogging or metal corrosion.

Using Tap Water

- 1) Make sure that the Water Tube is firmly connected to the Water Tube Connector of the Control Unit. Refer to “5-7 Connecting the Water Tube” for details.
- 2) Open the water valve of the dental unit and set the water pressure between 0.1 - 0.3MPa (1.0 - 3.0kgf/cm²). If the water pressure exceeds 0.3MPa, use a Water Pressure Reducing Valve (option) to adjust the water pressure to within the specified range. Refer to “20 Option Parts List” for details about the water pressure reducing valve.

7-2 Mounting the Tip

- 1) Check that the screw parts of the Handpiece and Tip are clean (Fig. 14).
- 2) Push the Tip into the distal end of the Handpiece and screw the Tip until it is lightly tightened (Fig. 15).

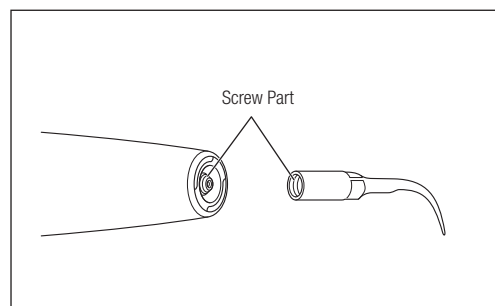
Fig. 14

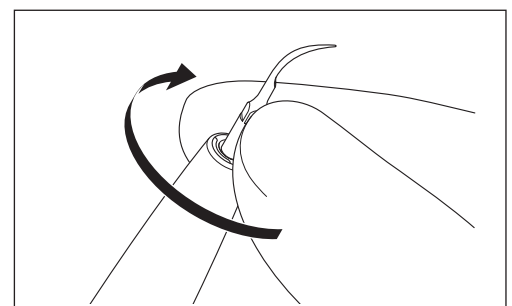
Fig. 15

- 3) Match the square shape of the Tip with the hole of the Tip Wrench, then insert the Tip into the hole. Rotate the Wrench until it makes a click sound and cannot rotate any further. During this procedure, hold the Handpiece so that the Handpiece Cord does not become twisted. When removing the Tip, rotate the Wrench in the direction of the “Loosen” arrow (Fig. 16).

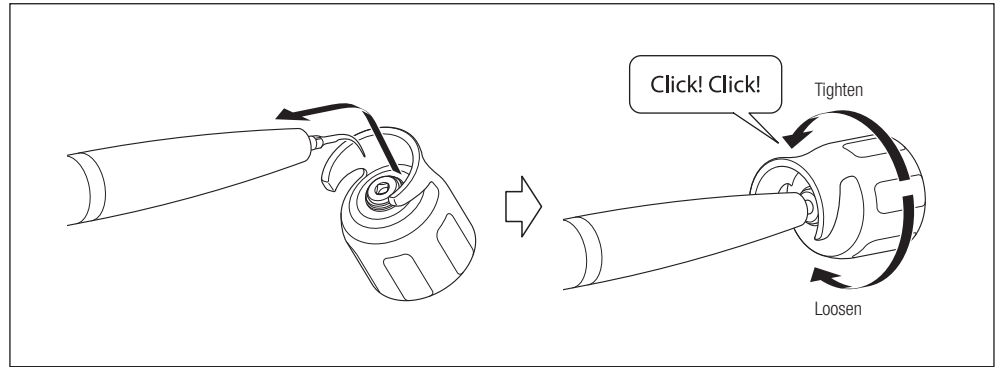
Fig. 16

WARNING

- Do not allow contact with gingival, mucosa and/or skin. Doing so may result in injury or burns.
- USE ONLY NSK genuine tips when using this product. Problems including Handpiece damage, failure and accidents resulting from the use of non-NSK tips are not covered in the warranty. The following types of issues may occur when using Non-NSK Tips:
 - Inappropriate vibration caused by incompatible screws.
 - Accidental ingestion of broken tips by the patient.
 - Damage or premature wear of the Handpiece screw

CAUTION

- Be careful not to twist the Handpiece Cord when tightening the Tip.
- Be careful when attaching or removing a Tip that is longer than the Tip Wrench, as there is a risk of injury from the point of the Tip.
- Do not use a Tip which has been damaged, bent or corroded as it may break during use.
- To avoid damaging the Tip thread or disengagement of the Tip, always make sure that the Tip is inserted at the correct angle and firmly tightened.
- If debris is found on the Tip thread, clean the Tip immediately. Failure to do so may result in noise or insufficient vibration.
- Do not attempt to sharpen or bend the Tip as doing so may result in the Tip breaking during use, noise or insufficient vibration.
- The Tip is a wear item. Worn Tips have an increased risk of noise, breakage or insufficient vibration. Replace the Tip if noise or insufficient vibration occurs.
- Use the supplied Tip Wrench to mount the Tip. Rotate the Wrench until it makes a click sound and cannot rotate any further. If the Tip is not tightened sufficiently, insufficient vibration or noise may occur.
- When mounting a Tip, always wear sterile surgical gloves and use a sterilized Tip, Handpiece and Tip Wrench.
- Ensure that the power setting does not exceed the recommended Power Level. Failure to do so may result in noise, tip breakage or injury to the patient.
- Do not use Tips that are not designed for removing metal or prosthetic crowns for this purpose. Doing so may result in the tip breaking and falling into the patient's mouth.
- Remove the Tip before attaching or removing the Handpiece or the Handpiece Cord to avoid to avoid injury with the Tip.
- If insufficient vibration is observed during operation, remove the Handpiece from the patient's mouth, and depress the Foot Control again. If this does not improve the condition, the Tip may have loosened during use. Check that the Tip is securely mounted.
- If the Tip becomes loose, makes noise, or does not vibrate sufficiently even though it has been tightened until it makes a click sound, the Tip Wrench may be worn out. The Tip Wrench is a wear item and should be replaced once a year.

■ Using the Tip Card

CAUTION • Tips are wear items. Use the supplied Tip Card to check the Tip condition and replace the Tip if necessary.

Place the neck of the Tip in the cut out of the Tip Card. Measure the length of the Tip and check the replacement timing (Fig. 17). To guarantee safe and effective use, it is recommended that Tips be replaced when the Tip meets the yellow line (1mm of wear).

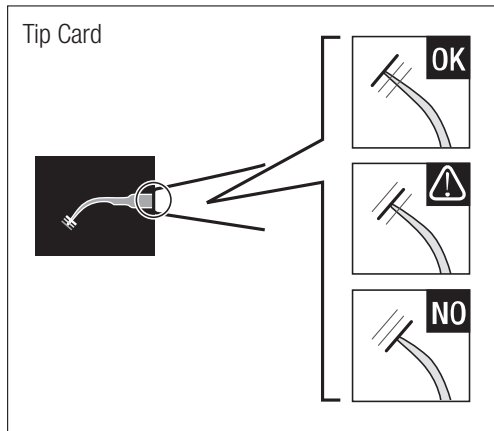
Fig. 17

CAUTION • Tips are consumables. The efficiency of dental scaling decreases approximately 25% when the top of the Tip wears 1mm and approximately 50% when it wears 2mm. In addition, the vibration condition changes owing to the wear, which may damage a patient's tooth surface. Periodically check the Tip wear condition using the Tip Card, and replace the Tip with a new one in good time.

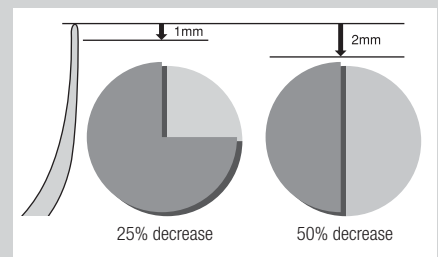
Fig. 18

7-3 Power Level Setting

- 1) Touch the Ultrasonic Mode Select Key to select the Mode. The selected Mode is displayed on the LCD Panel as P/E/G (Fig. 19).
- 2) Touch the Power Level Key (▲/▼) to set the Power Level. The Power Level Display and the Power Level Indicator increase or decrease in accordance with the selected level (Fig. 20).

CAUTION • Ensure the power setting does not exceed the recommended Power Level. (Refer to the supplied "Tip-Power Guide" or "Tip Guide" for the maximum power of each Tip.)

NOTICE • Touch and hold the Power Level Key to increase or decrease the Power Level continuously. An alarm is heard when the maximum or minimum level is reached.
 • To irrigate without vibrating the Tip, set the Power Level to "0 (zero)" and the Irrigation Volume to 1 bar or more.

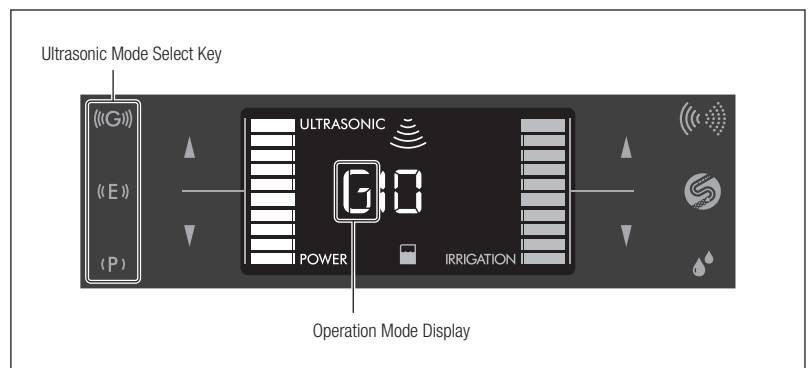
Fig. 19

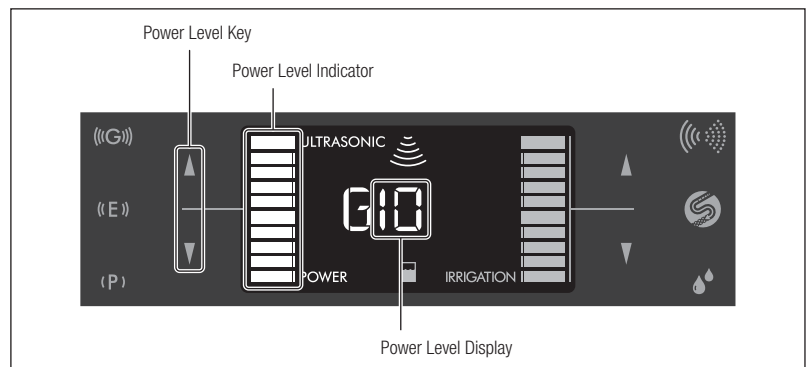
Fig. 20

7-4 Irrigation setting

Touch the Irrigation Mode Select Key to select the Irrigation Mode (Bottle or tap water). The selected Mode is displayed on the LCD Panel as an icon (Fig. 21).

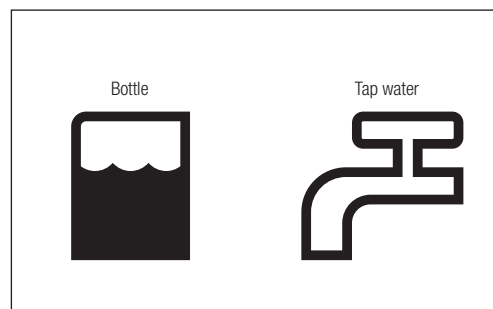
Fig. 21

7-5 Operation

- 1) Depress the desired button on the Foot Control. A “beep” sound is heard, the Tip starts vibrating and the LED on the Handpiece lights up.
- 2) Adjust the Irrigation Volume by touching the Irrigation Volume Keys (▲/▼) while checking the irrigation from the Tip.

CAUTION

- As a safety feature, this product does not operate when the power switch is turned on while the Foot Control is depressed. In this case, an alarm sound is heard and the Ultrasonic Mode Display (P/E/G) and “ULTRASONIC” blinks alternately. Release the Foot Control and depress it again.
- Always use irrigation. If there is insufficient irrigation the Handpiece will overheat and may injure the surface of the patient's tooth.
- Check that clean water is spraying adequately before use.
- Water may not spray smoothly if the Irrigation Volume is set to a low level. In this case, temporarily increase the Irrigation Volume temporarily and depress the Foot Control. Check that water is spraying smoothly from the Tip and adjust the Irrigation Volume again.
- Do not forcibly rotate the Handpiece Holder as doing so may damage the Holder.

NOTICE

- The LED of the Handpiece stays lit for about 5 seconds after the Foot Control is released (Afterglow function).
- Ultrasonic Mode (P/E/G), Power Level, Irrigation Mode settings and Irrigation Volume are stored after the power is turned off and the same settings are applied when the power is turned on next time (Last Memory function).
- If the Irrigation Volume is too low with the Irrigation Volume set to 10 or the Irrigation Volume is too high with the Irrigation Volume set to 1 while the Irrigation Mode is set to tap water, adjust the basic Irrigation Level. Refer to “11-3 Basic Water Level Setting” for details.
- A buzzing sound may be emitted, but this does not indicate a malfunction.

■ During the Handpiece operation :

You can: Adjust the Power Level and Irrigation Volume

You cannot: Change the Ultrasonic Mode or Irrigation Mode settings and perform Auto-Cleaning

7-6 After treatment

Perform maintenance after use on each patient. Refer to “9 Maintenance” for details.

CAUTION

- This product may malfunction if correct maintenance is not performed.

7-7 Protection Circuit

This product may overheat internally if the control unit is used at power 8 or higher in G mode for an extended amount of time. In this case, the Protection Circuit reduces the Power automatically (Power 7). Bar Graph Indicator flashes from 8 to 10 (Fig. 22). The indicator stops flashing when the protection circuit is released. However, in this case the Power Level does not increase automatically. Increase the Power Level manually if required.

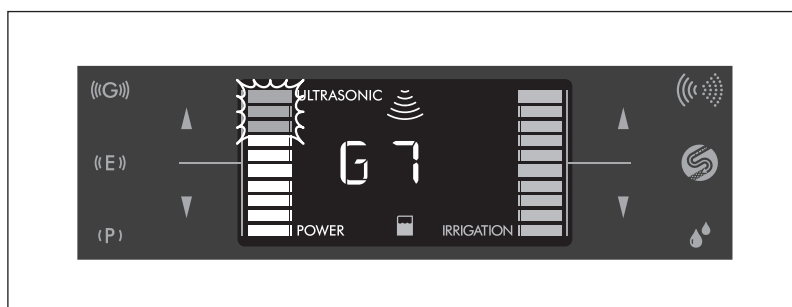
Fig. 22

NOTICE

- The Power Level cannot be increased to 8 or higher when the protection circuit has operated (while the bar graph indicator is flashing).
- The Bar Graph Indicator stops flashing when the Power Level is reduced below 7. It starts flashing again if the Power Level is increased to 7 while the Protection Circuit function is active.

8 Using the Powder Mode

The optional Perio Kit must be purchased in order to use the Perio Mode. Refer to “20 Option Parts List” for details about the Perio Kit.

8-1 Water System Setup

- 1) Make sure that the Water Tube is firmly connected to the Water Tube Connector of the Control Unit. Refer to “5-7 Connecting the Water Tube” for details.
- 2) Open the water valve of the dental unit and set the water pressure between 0.1 - 0.3MPa (1.0 - 3.0kgf/cm²). If the water pressure exceeds 0.3MPa, use a Water Pressure Reducing Valve (option) to adjust the water pressure to within the specified range. Refer to “20 Option Parts List” for details about the water pressure reducing valve.

- Only use drinkable water for irrigation.

CAUTION

8-2 Powder Preparation

- 1) Select the Chamber and powder to be used with this Mode. Fill the Chamber with powder up to the “MAX” line (approximately 80g of FLASH pearl or approximately 50g of Perio-Mate powder) (Fig. 23).

Mode	Chamber	Powder
Prophy Mode	For Prophy use	FLASH pearl
Perio Mode	For Perio use	Perio-Mate powder

- 2) Close the Chamber Cap securely until it fits the Chamber completely and insert the Chamber straight to the Control Unit (Fig. 24). The type of inserted Chamber is displayed on the LCD Panel as one of the following icons (Fig. 25).

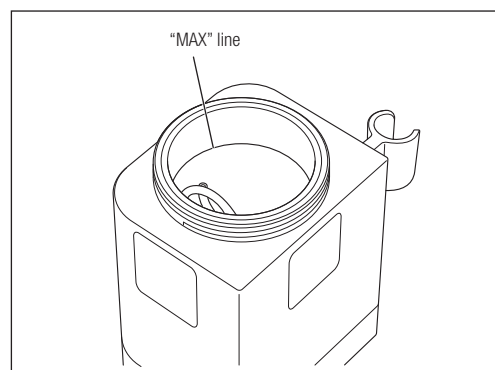
Fig. 23

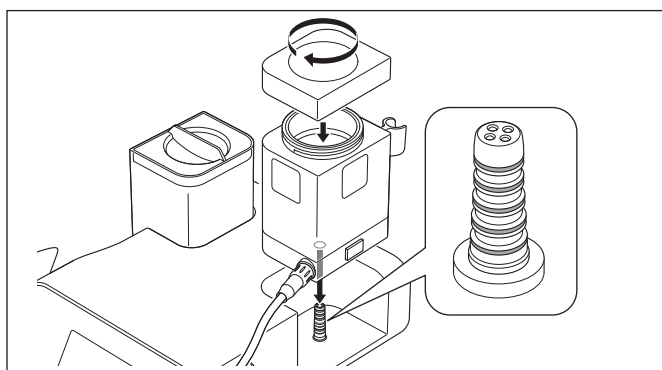
Fig. 24

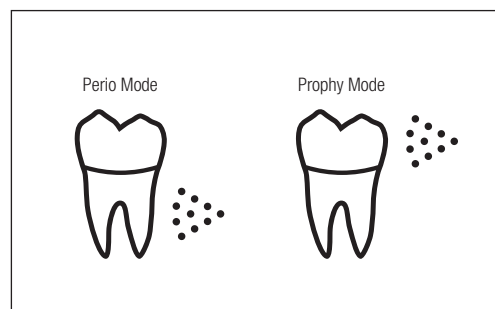
Fig. 25

WARNING

- Use the designated powder only. Using other powders may lead to product failure, damage to the tooth or may cause emphysema. In addition, it may also cause performance reduction of the product. We cannot accept any liability for failures caused by using other powders.

CAUTION

- Securely attach the Chamber Cap to prevent the chamber cap from being blown off, which may lead to an injury (Fig.26).
- Before inserting the Chamber into Control Unit, make sure the O-Rings of the Chamber Base Connector are not damaged or torn (Fig.24). Damaged or torn O-Rings could lead to poor operation or malfunction.
- Insert the Chamber into the Control Unit straight. Inserting at an angle could damage the O-Rings, leading to malfunction.
- If too much cleaning powder is put into the Chamber, the correct amount of powder may not be sprayed immediately after starting use.
- Do not use powder that has been stored in an open package or leftover powder. Using such powder may cause clogging of the Nozzle. Do not mix leftover powder with new powder.
- Before inserting the Chamber, check the Chamber Base Connector for water, etc., and wipe it clean if it is wet. Failing to do so may cause powder clogging.
- Do not use discolored powder.
- Always securely close the powder container lid. Failure to do so will attract moisture which may cause clogging.
- If powder adheres to the Chamber and/or the Chamber Cap, clean it immediately or the Cap will become difficult to close.
- Do not invert the Chamber while it contains powder inside. Inverting the Chamber could lead to powder clogging.

Fig. 26

CAUTION

- As a safety feature, this product does not operate if the Foot Control is depressed when the Chamber is mounted incorrectly. An alarm sounds heard and the tooth icon and “POWDER” flash alternately. Mount the Chamber correctly.
- The Perio-Mate Powder (option) scatters easily because it is made of very fine particles. Fill the Chamber slowly to prevent the powder from scattering, and be careful not to swallow the powder or allow it to get into eyes.
- FLASH pearl is almost insoluble in water. To avoid clogging of the pipes by using FLASH pearl, periodical cleaning and maintenance of the vacuum hose, saliva ejector and separator are necessary.

8-3 Attaching/removing the Nozzle Tip

8-3-1 Attaching the Nozzle Tip

Attach the Nozzle Tip (option) securely and firmly to the Nozzle of the Powder Handpiece (for Perio use) (Fig. 27).

WARNING

- Do not use a deformed or damaged Nozzle Tip. It may detach during use.

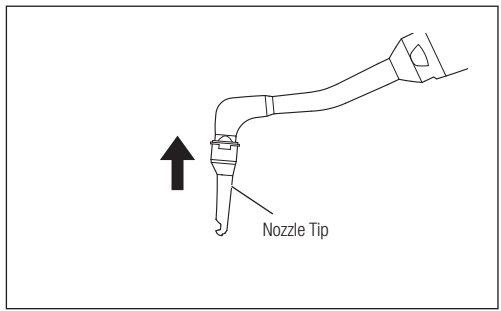
Fig. 27

CAUTION

- Be sure to attach the Nozzle Tip securely and firmly as illustrated (Fig. 28). If the Tip is attached incorrectly, it will fall off and may be accidentally ingested by the patient.
- When attaching a Nozzle Tip to the Powder Handpiece, always wear sterile surgical gloves and make sure that the Powder Handpiece is sterilized.

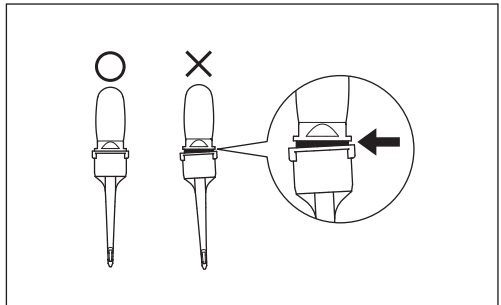
Fig. 28

8-3-2 Removing the Nozzle Tip

Use the Perio-Mate Nozzle Tip Remover (option) to remove the Nozzle Tip (Fig. 29) and dispose of the Nozzle Tip when removed.

WARNING

- The Nozzle Tip can only be used once. Do not reuse or sterilize the Nozzle Tip as it may be damaged or contaminated. If the Nozzle Tip is reused, it may not operate correctly due to deformation during removal, or it may detach during use resulting in accidental ingestion by the patient. The same applies to a Tip that has been accidentally removed.

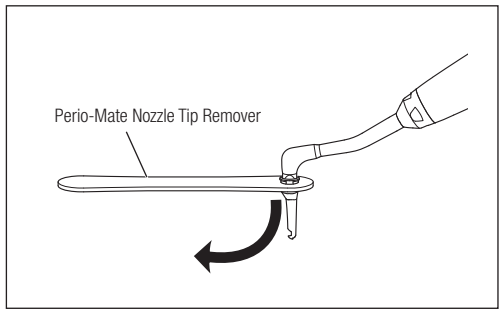
Fig. 29

8-4 Power Level Setting

Touch the Power Level Keys (▲/▼) to set the Power Level. The Power Level Display and the Power Level Indicator increase or decrease in accordance with the selected Power Level (Fig. 30).

NOTICE

- If you specify this setting without attaching the Chamber, the Power Level when using the Prophy mode is set.

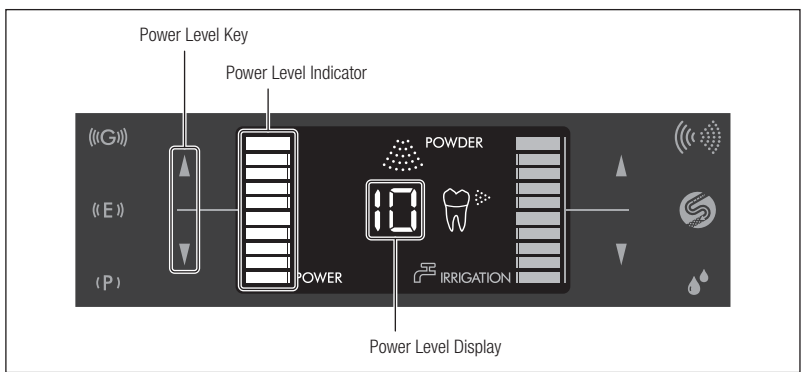
Fig. 30

8-5 Irrigation setting

- 1) Touch the Irrigation Volume Keys (▲/▼) to adjust the Irrigation Volume.
- 2) Depress the Foot Control to spray powder into a spittoon, etc., to confirm that powder, air and water are appropriately sprayed.
- 3) Adjust the Irrigation Volume while checking the irrigation from the Nozzle.

NOTICE

- If the Irrigation Volume is too low with the Irrigation Volume set to 10 or the Irrigation Volume is too high with the Irrigation Volume set to 1, adjust the basic Irrigation Level. Refer to “11-3 Basic Water Level Setting” for details.
- If you perform adjustment without attaching the Chamber, the Irrigation Volume when using the Prophy mode is set.

8-6 Operation

Powder, water, and air are supplied when the Foot Control button D is depressed. Water and air are supplied when the Foot Control button B is depressed. Observe the operating instructions explained in “8-6-1 Using Prophy Mode” and “8-6-2 Using Perio Mode.”

WARNING

- Never spray powder toward a human body including the operator and the assistant other than to the tooth surface.
- Stop use immediately if the hose swells or air or powder leaks from connections between the Chamber and Hose during use.
- Spray water while using this product.
- Be sure to use a vacuum tube (and a saliva discharge tube together if necessary) to prevent the patient from ingesting a large amount of powder. Avoid continuous use over long periods and allow the patient to rinse out the mouth whenever necessary.

CAUTION

- As a safety feature, this product will not switch on when the Power Switch is turned to ON while the Foot Control is depressed. An alarm sound is heard and the tooth icon and “POWDER” flash alternately. Release the Foot Control then depress it again.
- Before treatment, adjust the Power Level and the Irrigation Level before using on the patient.
- Reduce the Power Level when treating around marginal gingiva.
- Do not spray vertically onto the tooth surface.
- Maintain the distance between the tip of the Nozzle of the Handpiece to the tooth surface as explained in “8-6-1 Using Prophy Mode” and “8-6-2 Using Perio Mode.”
- Do not rotate the Handpiece Holder forcibly as it may damage the Holder.

NOTICE • A buzzing sound may be emitted, but this does not indicate a malfunction.

8-6-1 Using Prophy Mode

Keep the Nozzle at a distance of 3 - 5mm from the tooth surface and at an angle of 10 - 60 degrees (Fig. 31).

WARNING

- Never point toward soft tissue, gum, or the subgingival area.

CAUTION

- Spray from the root to the incisal edge of the tooth.

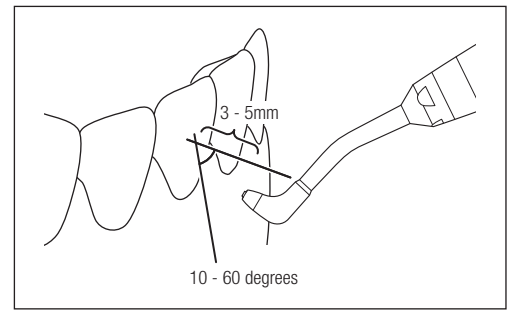
Fig. 31

8-6-2 Using Perio Mode

■ Using with the Nozzle Tip

Insert the Nozzle Tip deeper than 3mm into the periodontal pocket (Fig. 32). Powder may not be sprayed efficiently into the periodontal pocket if the depth is less than 3mm.

■ Using without the Nozzle Tip

Remove the Nozzle Tip from the Nozzle of the Handpiece to remove plaque at the supragingival area and gingival margin, or to remove biofilm and light stains. Keep the Nozzle at a distance of 5 - 10mm from the tooth surface and at an angle of 30 - 60 degrees (Fig. 33). Never point the Nozzle directly toward the oral mucous membrane or in periodontal pockets.

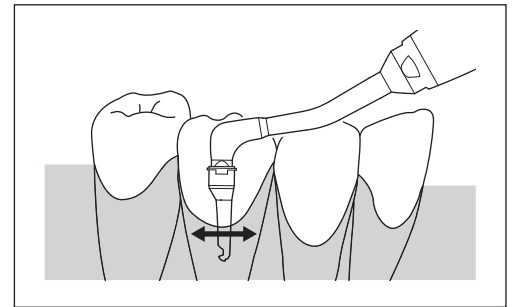
Fig. 32

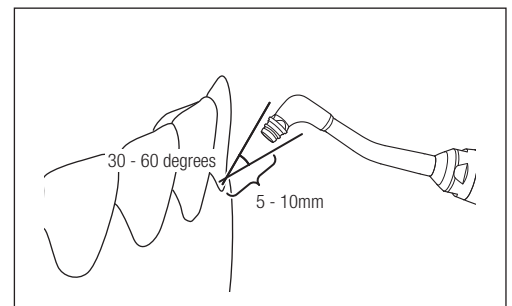
Fig. 33

- WARNING**  • When using without the Nozzle Tip, never point toward the oral mucous membrane.
- Do not use this product in an area immediately after scaling the tooth surface in a periodontal pocket. Doing so may cause bacteremia.
 - Do not insert the Nozzle Tip more than once into the same periodontal pocket during the same treatment.
 - Powder may be sprayed for a maximum of 5 seconds in the same periodontal pocket (maximum 20 seconds for one tooth). Never spray powder for longer than the specified time.
 - Do not force the Nozzle Tip into a periodontal pocket.
 - Do not spray powder while the Nozzle Tip end is located at the bottom of a periodontal pocket. Doing so may cause emphysema.
 - Do not use the Nozzle Tip in any places other than periodontal pockets.
 - Setting the Power Level too high may cause emphysema. Adjust the Power Level to an adequate level while checking the patient's condition.

- CAUTION**  • The alarm sound is heard every 5 seconds when using Perio Mode.
- In Perio Mode, the Power Level is automatically reduced to a weaker level compared to the same setting for Prophy Mode.

8-6-3 During Handpiece operation

You can: Adjust the Power Level and Water Volume

You cannot: Change Irrigation Mode settings and perform Auto-Cleaning

8-6-4 Releasing residual pressure

The Chamber Cap cannot be opened after operation due to buildup of air pressure inside the Chamber increasing during operation.

Release the pressure remaining in the Chamber before putting powder in the Chamber or removing the Chamber from the Control Unit.

- CAUTION**  • The remaining powder and water comes out of the Handpiece while releasing the residual pressure. Prepare a container such as a cup to catch the powder and water.

- 1) Take the Powder Handpiece out of the Handpiece Holder.
- 2) Place the Handpiece into the container to catch powder and water.
- 3) Touch and hold the Auto-Cleaning Key for about 1 second. Beeps are heard and the residual pressure starts to be released.
- 4) The operation stops when all the residual pressure is released from the Chamber.

8-7 After treatment

Perform maintenance after use on each patient. Refer to "9 Maintenance" for details.

If the Nozzle Tip (option) is attached to the Nozzle, remove and dispose of it before maintenance.

- CAUTION**  • Failure to perform maintenance may result in malfunction.

- NOTICE** • The Power Level and Irrigation Mode settings are stored after the power is turned off (Last Memory function).

9 Maintenance

Perform the following maintenance after use on each patient.

 • Failure to perform maintenance may result in malfunction.
CAUTION

ULTRASONIC

9-1 Ultrasonic Handpiece

9-1-1 Auto Cleaning

 • Water comes out of the Handpiece during Auto-Cleaning. Perform cleaning after placing the Handpiece into a cup.
CAUTION

- 1) Take the Ultrasonic Handpiece out of the Handpiece Holder and remove the Tip from the Handpiece.
- 2) Remove the Irrigation Bottle and wash inside with running water. When tap water is selected for the Irrigation Mode, open the water valve of the irrigation source.
- 3) Fill more than half of the Irrigation Bottle with distilled water or purified water. (Do not use saline).
- 4) Close the Cap securely.
- 5) Insert the Bottle straight into the Irrigation Bottle Base Connector. Water may leak if the Bottle is not inserted firmly.
- 6) Place the tip of the Handpiece into the container to catch water.
- 7) Touch and hold the Auto-Cleaning Key for about 3 seconds to start Auto-Cleaning. Auto-Cleaning takes about 45 seconds. "AC" is displayed on the LCD Panel during Auto-Cleaning and the approximate remaining time is displayed in the Bar Graph Indicator (Fig. 34).
- 8) When Auto-Cleaning is finished, the Control Unit returns to the settings prior to cleaning.
- 9) Remove the Irrigation Bottle. Wash the Irrigation Bottle and Cap with running water and allow them to dry.

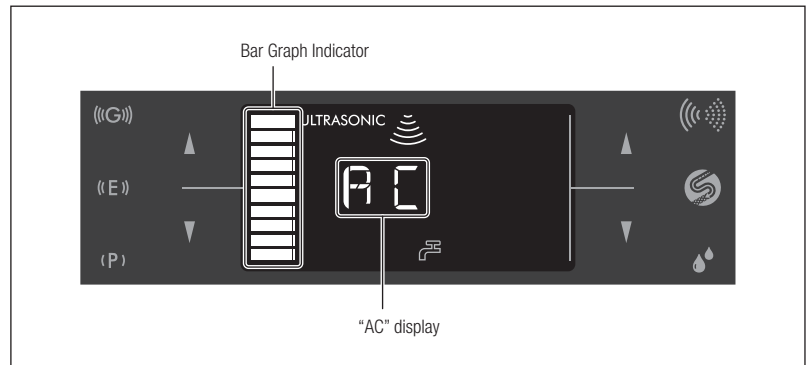
Fig. 34

To stop Auto-Cleaning, touch the Auto-Cleaning Key again.

 • After each use, perform Auto-Cleaning (cleaning of Irrigation Tube). If the system is not cleaned, the inside of the Irrigation Tube may become clogged or some metal parts may rust.
CAUTION

- Use distilled water or purified water to fill the Irrigation Bottle. Using impure water may cause clogging of the Irrigation Tube.
- Make sure the Handpiece and Handpiece Cord are firmly attached.

NOTICE • During Auto-Cleaning, the Handpiece LED does not light.

9-1-2 Cleaning the Handpiece

- 1) Remove any debris from the product. Do NOT use a wire brush.
- 2) Wipe clean with a cloth soaked in ethanol, or ethanol with isopropyl alcohol.

 This icon denotes that the product can be washed using a Thermo Disinfector.

Operation only confirmed with the GETINGE WD46-4 model.
Refer to the Thermo-Disinfector manual.

 • Be careful not to hit the tip of the Handpiece against anything while washing it as it may damage the Glass Rod.
CAUTION

- After cleaning with the Thermo Disinfector, dry the Handpiece thoroughly and remove water from the inside.
- Never use solvents such as benzine and thinner for cleaning.
- When cleaning, use ethanol, or ethanol with isopropyl alcohol to wipe the surface of the handpiece. Use of chemicals other than those may cause the handpiece to discolor, crack, etc.

9-1-3 Cleaning (Optic)

Wipe clean the Glass Rod tip with an alcohol-immersed cotton swab. Remove all debris (Fig. 35).

- Do NOT use a sharp tool to clean the Glass Rod. Doing so may damage the glass and reduce light transmission.

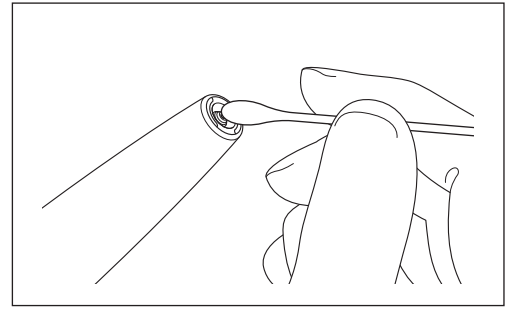
Fig. 35

9-1-4 Cleaning the Handpiece cord

Remove the handpiece after use on each patient and clean it as described below.

- Wipe the surface of the handpiece cord with a cloth soaked in alcohol.
- Carefully wipe the handpiece cord plug with an alcohol-immersed cotton swab. If it is difficult to use a cotton swab, carefully wipe with a towelette wound around a thin stick-shaped object.

- Do not use a sharp pointed stick or push the terminal part when cleaning the handpiece cord plug. Doing so may cause damage, resulting in contact failure (Fig. 36).

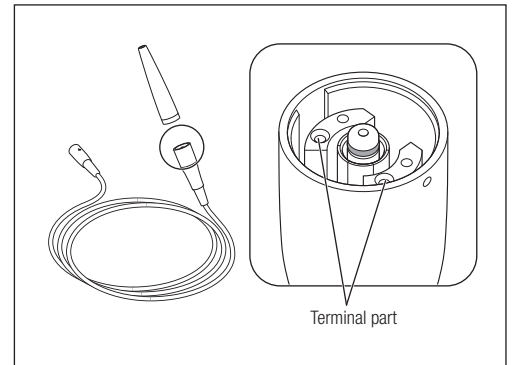
Fig. 36

9-2 Powder Handpiece

9-2-1 Auto-Cleaning

- Water comes out of the Handpiece during Auto-Cleaning. Perform cleaning after placing the Handpiece into a cup.

- Take the Powder Handpiece out of the Handpiece Holder. For the Powder Handpiece (for Perio use), remove the Nozzle Tip from the Handpiece.
- Place the tip of the Handpiece into the container to catch water.
- Touch and hold the Auto-Cleaning Key for about 3 seconds to start Auto-Cleaning.
- Water and air are sprayed first to wash off the remaining powder.
- After a while, only air is sprayed to release the remaining water.
- Auto-Cleaning takes about 45 seconds. "AC" is displayed on the LCD Panel during Auto-Cleaning and the approximate remaining time is displayed by the Bar Graph Indicator (Fig. 37).
- When Auto-Cleaning is completed, the Control Unit returns to the settings prior to cleaning.

To stop Auto-Cleaning, touch the Auto-Cleaning Key again.

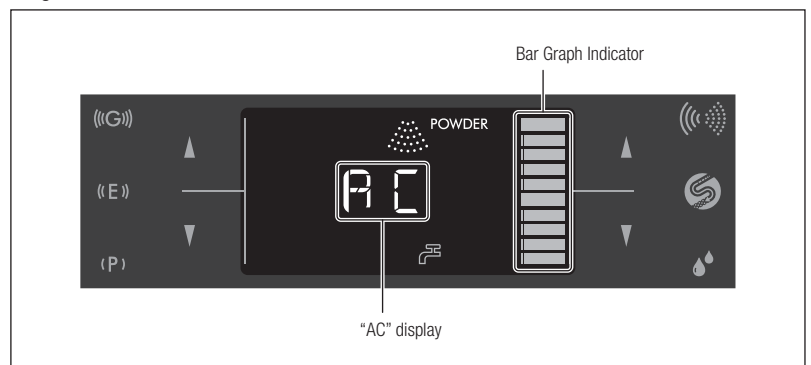
Fig. 37

9-2-2 Cleaning (Handpiece)

- 1) Remove the Handpiece from the Hose.
- 2) Remove debris from the product. Do NOT use a wire brush.
- 3) Use the supplied Cleaning Brush to remove powder from the interior section where the Handpiece was connected (Fig. 38).
- 4) Wipe clean with a cloth soaked in alcohol.

 This icon denotes that the product can be washed using a Thermo Disinfector.

Operation only confirmed with GETINGE WD46-4 model.
Refer to the Thermo-Disinfector manual.

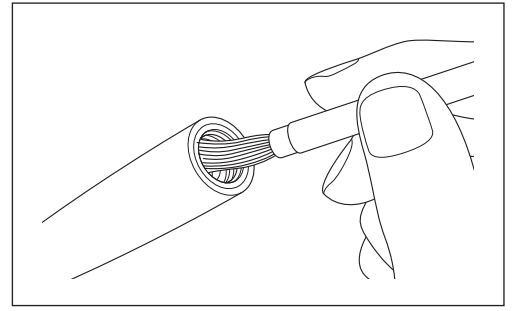
Fig. 38

 • Be sure to clean the inside of the Handpiece after each use. Powder remaining in the Handpiece may solidify, causing clogging.
WARNING

 • After washing with a Thermo-Disinfector, dry the product until all internal moisture is thoroughly removed.
• Never use solvents such as benzine or thinner to clean the product.
CAUTION

9-2-3 Ultrasonic Cleaning

 • Be sure to perform Ultrasonic Cleaning at least once a week to prevent powder clogging. Perform Ultrasonic Cleaning before Auto-Cleaning or after you have tried the remedies described in "14 Troubleshooting."
CAUTION

- 1) Pass through the supplied Ring Wrench from the Nozzle end, loosen the Nozzle, and remove the Nozzle from the Handpiece (Fig. 39).
- 2) Perform Ultrasonic Cleaning with warm water (30 - 50°C) on the Handpiece and Nozzle.
- 3) After cleaning, use air to dry all of the parts.
- 4) Reassemble the Handpiece following the reverse of procedures in step 1.
- 5) Connect the Handpiece and Hose and perform Auto-Cleaning.

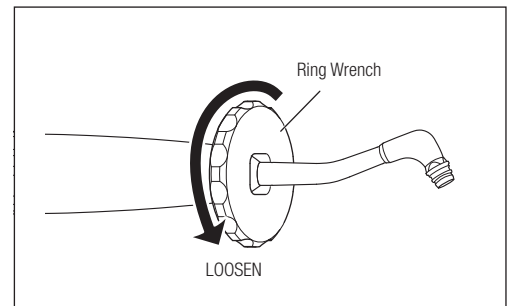
Fig. 39

 • When attaching the Nozzle, lightly screw the Nozzle into the Handpiece by hand at first and then securely tighten it with the supplied Ring Wrench.
• After Ultrasonic Cleaning, perform Auto-Cleaning to remove any remaining water from the inside, and allow the parts to dry sufficiently before use.
• To prevent the Nozzle from being deformed, always use the supplied Ring Wrench when attaching or removing the Nozzle. The nozzle may become deformed if you attach it while holding the end of the nozzle.
• Ultrasonic Handpiece cannot be cleaned by Ultrasonic Cleaning.
CAUTION

9-3 Sterilization

This product must be autoclaved. Autoclave sterilization is required after use on each patient as described below.

Autoclavable items: Ultrasonic Tip, Ultrasonic Handpiece, Powder Handpiece (for Prophy use), Powder Handpiece (for Perio use, option), Tip Wrench, Sterilization Case, Perio-Mate Nozzle Tip Remover (option)

- 1) Insert the items into an autoclave pouch. Seal the pouch.
- 2) Autoclave under the conditions below.
Autoclave for more than 20 min. at 121°C, 15 min. at 132°C or 3 min. at 134°C.
- 3) Keep the items in the autoclave pouch to maintain sterility until it is used.

■ Sterilization Case

The Ultrasonic Handpiece, Ultrasonic Tip and Tip Wrench can be sterilized together using a Sterilization Case. Four Tip Wrenches and Tips can be set at once (Fig. 40).

- 1) Remove the Tip after use. (Refer to “7-2 Mounting the Tip.”) Tip is set inside the Tip Wrench.
- 2) Set the Tip Wrench with Tip attached into the Sterilization Case.
- 3) Set the Ultrasonic Handpiece into the Sterilization Case.

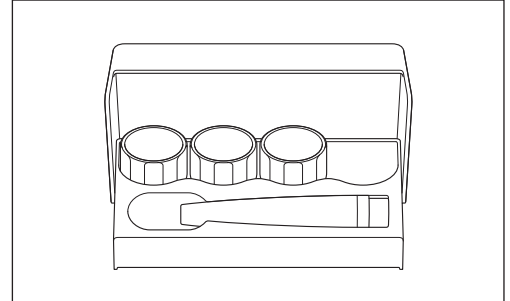
Fig. 40

CAUTION

- Sterilization Case cannot be used for the Powder Handpiece.
- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- If the sterilizer chamber temperature may exceed 135°C during the dry cycle then delete the dry cycle.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.
- Do not autoclave the Handpiece Cord and the O-Ring. Disinfect the Handpiece Cord with a cloth soaked in alcohol after every patient.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the product.

NOTICE

- NSK recommends Class B sterilizers as stated in EN13060.
- Repeated autoclaving may cause the ultrasonic handpiece to become discolored due to heat. However, this is due to properties of the product and is not a problem in terms of quality.

9-4 Cleaning the Chamber and Hose

- 1) Perform Auto-Cleaning before cleaning the Chamber to release residual powder and water from the inside of the Handpiece and Handpiece Hose.
- 2) Release the pressure remaining in the Chamber. (Refer to “8-6-4 Releasing residual pressure”.)
- 3) Push the Lock Button to remove the Chamber. Then pull up the Chamber straight (Fig. 41).
- 4) Open the Chamber Cap and remove the residual powder.
- 5) Loosen the Handpiece Hose Connector nut and remove the Hose from the Chamber.
- 6) Remove the residual powder from the Chamber using air, etc.
- 7) Wipe the Hose to remove powder, etc.
- 8) Use the supplied Cleaning Brush to remove powder from the Connectors on both ends of the Handpiece Hose and the Handpiece Hose Connector Socket of the Chamber.

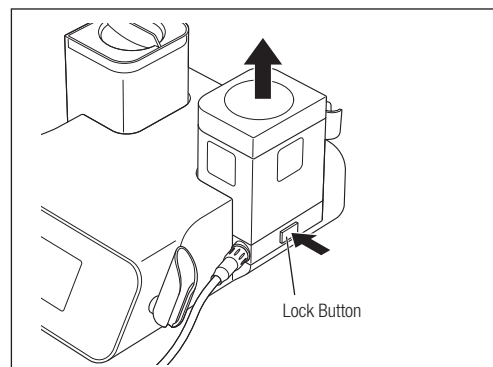
Fig. 41

WARNING

- Medicinal solution, solvents, etc. adhering to the Chamber or the Chamber Cap may cause damage.
- Be sure to clean the Chamber after each use. Powder remaining in the Chamber may solidify and cause clogging.
- Never use the Chamber Cap if it is cracked or scratched. Using in this condition may result in damage or injury from broken pieces.

CAUTION

- If a failure occurs in the Chamber, be sure to contact your Authorized NSK Dealer for repairs.
- After use, always thoroughly remove powder from the inside of the Chamber and clean the Chamber before storage.
- Do not wash the Chamber. Doing so may cause the powder to solidify, resulting in clogging.
- Perform the operation in “8-6-4 Releasing residual pressure” before removing the Chamber. If you do not do so, the Chamber Cap cannot be opened. The Chamber Cap may pop off if you try to force it open.

■ Releasing the Lock Button

If the Chamber cannot be removed because the Lock Button cannot be pressed, perform the following operation to release the Lock Button and remove the Chamber, and then contact your Authorized NSK Dealer.

- 1) Insert the included Unlocking Tool into the hole on the base of the Control Unit and push it in all the way. (Fig. 42)
The Lock Button is released.
- 2) Remove the Unlocking Tool.
- 3) Press the Lock Button to remove the Chamber.
If the Lock Button is still not released, contact your Authorized NSK Dealer.

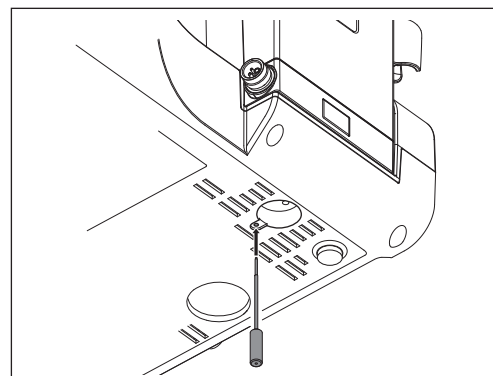
Fig. 42

CAUTION

- Do not turn the Control Unit upside-down when releasing the Lock Button. Failing to do so may cause powder to get clogged.
- Do not twist the Unlocking Tool when inserting it.

9-5 Cleaning the Control Unit and Foot Control

- 1) Turn OFF the power.
- 2) Disconnect the AC power cord from the wall socket.
- 3) Wipe off dirt and powder with a damp cloth and then wipe the surface with a cloth soaked in alcohol.

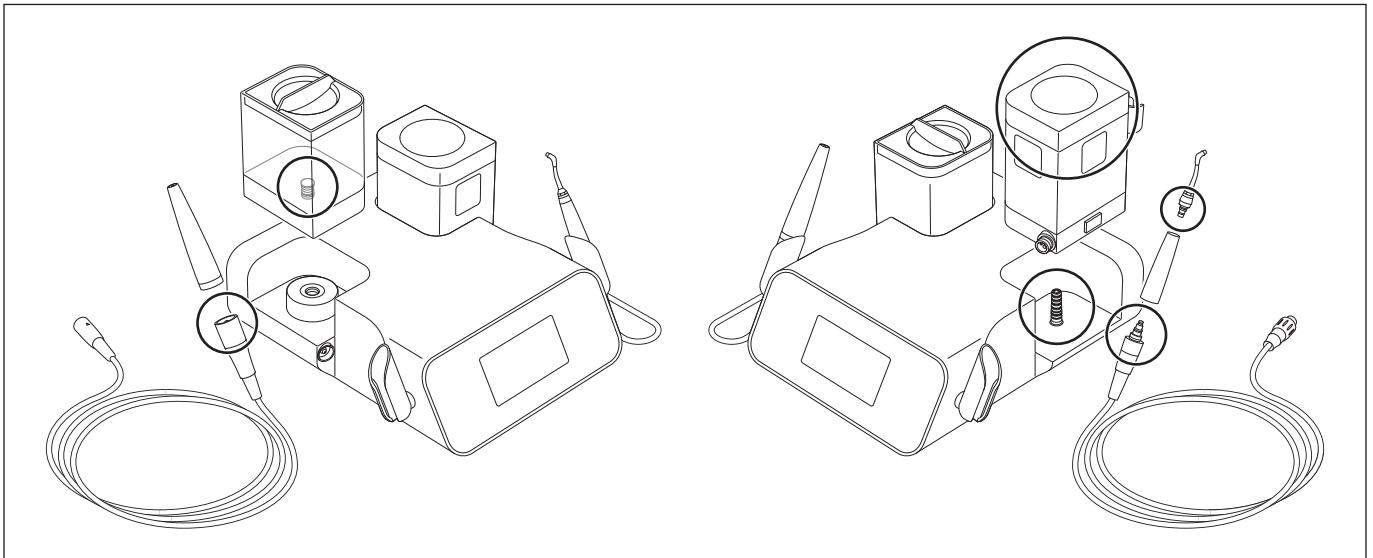
CAUTION

- Never use solvents such as benzine or thinner for cleaning.

10 Care and Maintenance

10-1 Replacing the O-rings

Replace the O-rings when symptoms such as leakage of water or air are observed.

ULTRASONIC

10-1-1 Handpiece Cord

- 1) Remove the Handpiece Cord from the Control Unit.
 - 2) Remove the O-ring to be replaced using a needle, etc (Fig. 43).
 - 3) Place the new O-ring in the groove from where the old O-ring was removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.
Order Code: Y1003546

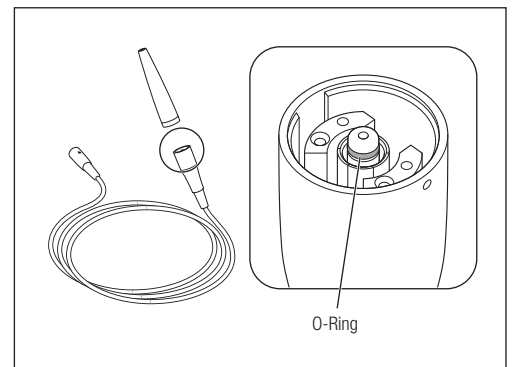
Fig. 43

POWDER PROPHY POWDER PERIO

10-1-2 Handpiece Hose (for Prophy use and Perio use)

- 1) Remove the Hose from the Powder Handpiece and Chamber.
 - 2) Remove the O-rings from the Handpiece Hose Plug using a needle, etc. If there is powder in the O-ring grooves, clean the groove before inserting the new O-ring.
 - 3) Place the new O-rings in the grooves from where the old O-rings were removed.
 - 4) Apply a drop of the supplied turbine oil and rub it in with your finger (Fig. 44).
- * Refer to "21 Spare Parts List" for details about replacement O-rings.
Order Code: Y1003500

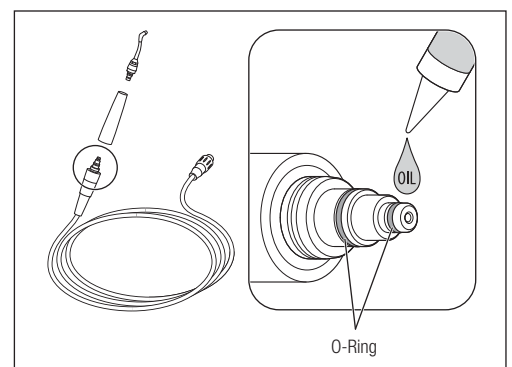
Fig. 44

CAUTION

- Failure to lubricate the O-ring section on the Handpiece side may cause damage to the Joint section, resulting in difficulty in rotating the Handpiece connection. It may also cause water leakage.

10-1-3 Nozzle (for Prophy use and Perio use)

- 1) Pass through the supplied Ring Wrench from the Nozzle end, loosen the Nozzle, and remove the Nozzle from the Handpiece.
 - 2) Remove the O-rings from the root of the Nozzle using a needle, etc. (Fig. 45). If there is powder in the O-ring grooves, clean the groove before inserting the new O-ring.
 - 3) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.
Order Code: Y1003501

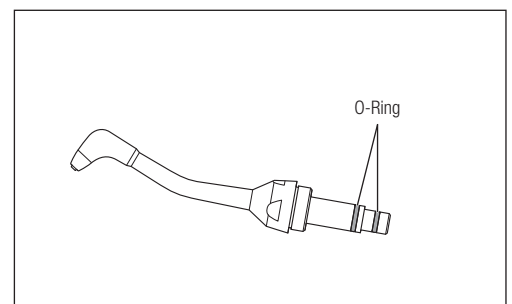
Fig. 45

10-1-4 Irrigation Bottle

- 1) Remove the O-rings from the Irrigation Bottle Joint using a needle, etc.
 - 2) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.
Order Code: Y1003544

- The O-rings are of different thicknesses. Be sure to place them in the correct groove (Fig. 46).

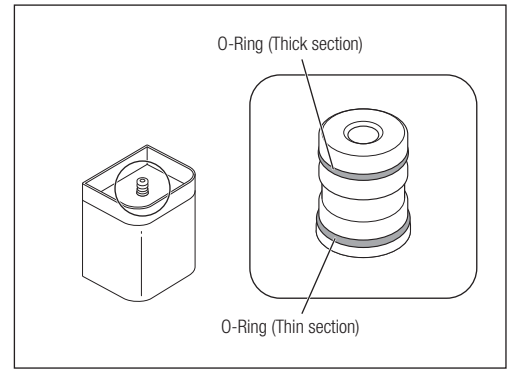
Fig. 46

10-1-5 Chamber Base Connector

- 1) Remove the O-rings from the Chamber Base Connector using a needle, etc. (Fig. 46).
 - 2) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.
Order Code: Y1003435

- Since the size of the O-Rings differ, make sure to replace them in the correct location (Fig. 47).

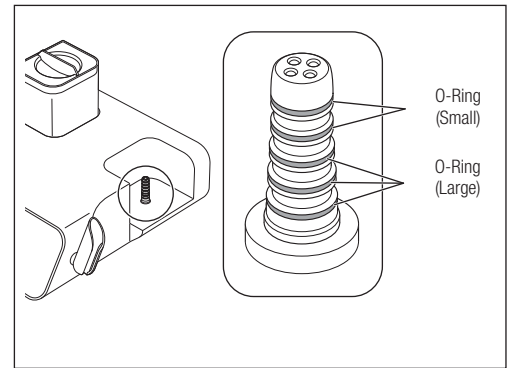
Fig. 47

10-1-6 Chamber Cap

- 1) Remove the Chamber Cap.
 - 2) Remove the O-ring from the inside of the Chamber Cap using a needle, etc. (Fig. 48).
 - 3) Place the new O-ring in the groove from where the old O-ring was removed.
- Order Code: Y1003545

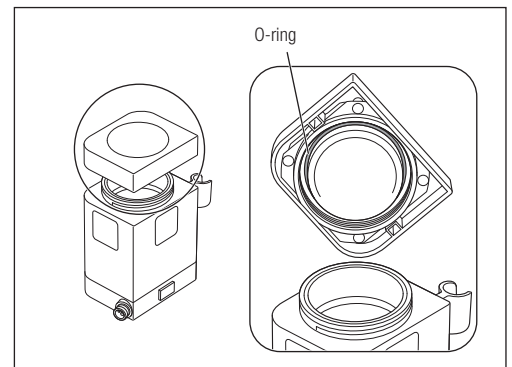
Fig. 48

10-2 Changing the Irrigation Pump

- 1) Remove the Irrigation Bottle, the Chamber, Power Cord, Handpiece Cord and Foot Control from the Control Unit.
- 2) Turn over the Control Unit. Use a cross-head screwdriver to remove the screw on the Irrigation Pump Cover and remove the Cover. (Fig. 49).
- 3) Remove two Irrigation Tubes from the Control Unit (Fig. 50).
- 4) Remove the Connector Ring from the Irrigation Tube. Do not dispose of it. You can use the Ring on the replacement Irrigation Pump (Fig. 51).
- 5) Turn the Irrigation Pump counterclockwise until it clicks, and then pull it out (Fig. 52).
- 6) Mount the Connector Rings in the Irrigation Tubes of the new Irrigation Pump (option). Make sure the Ring is oriented correctly (Fig. 53).
- 7) Align the new Irrigation Pump (option) with the Drive Shaft. Turn clockwise until it clicks (Fig. 54).
- 8) Mount the Irrigation Tube by following the removal procedure in reverse. Press the Connector Ring firmly into the Control Unit until it is fully pressed on (Fig. 50).
- 9) Align the Irrigation Pump Cover Hook and hole on the Control Unit. Mount the Cover.

* For details on the Optional Irrigation Pump, refer to "21 Spare Parts List."

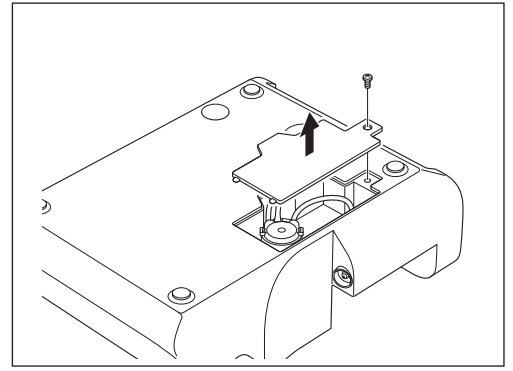
Fig. 49

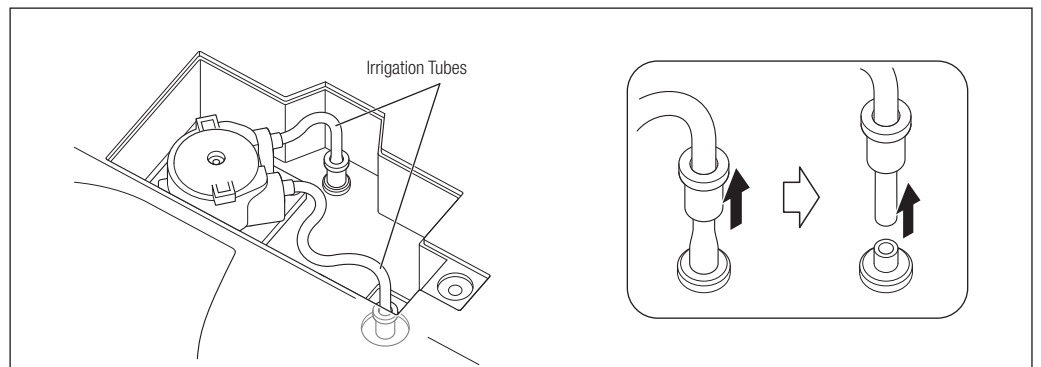
Fig. 50

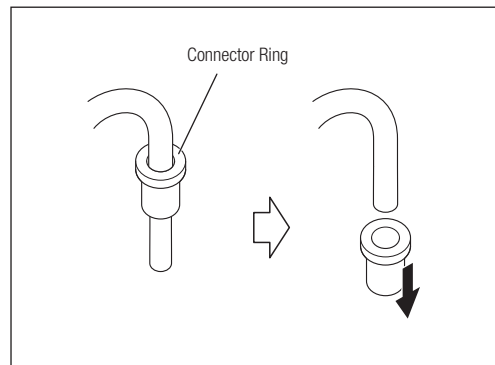
Fig. 51

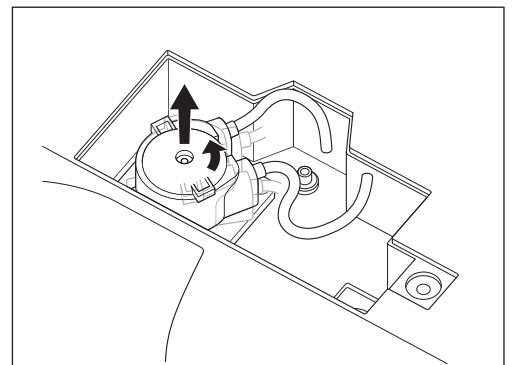
Fig. 52

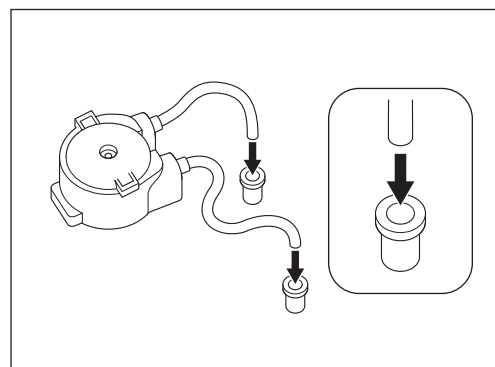
Fig. 53

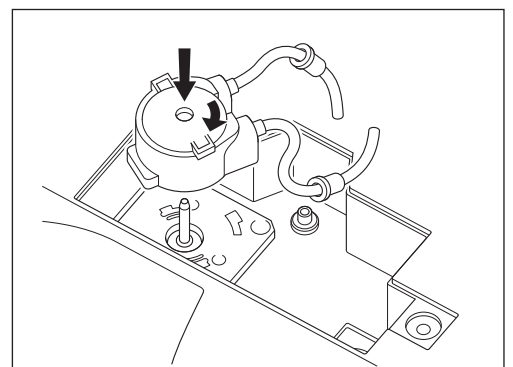
Fig. 54

CAUTION

- Before replacing the Irrigation Pump, wipe off excess water from the pump and Drive Shaft. Wet drive shaft and rollers can be slippery and cause improper operation.
- Wipe dirt and water off the Drive Shaft from the bottom up (Fig. 55).
- Insert the replacement Irrigation Pump straight into the Drive Shaft in a slow and gentle manner to prevent damaging the pump rollers.
- After replacing the Irrigation Pump, run this product for about 10 seconds with Irrigation Mode set to Bottle and Irrigation Volume set to maximum so the Irrigation Tube adapts to the new Pump.
- Ensure there are no kinks or twists in the Irrigation tube. If tube is set incorrectly, Irrigation Water may not be supplied.
- Make sure that the Tube is not caught under the Irrigation Pump Cover when closing the Irrigation Pump Cover.
- Only use the dedicated Irrigation Pump for Varios Combi Pro. Other pumps cannot be used.

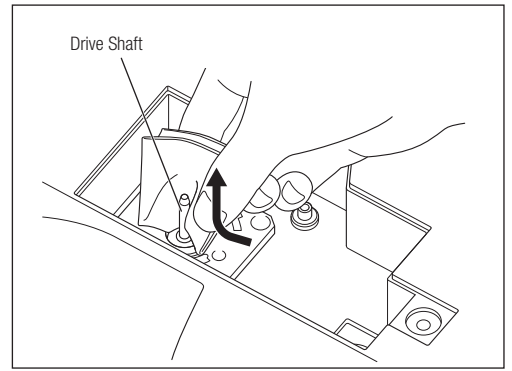
Fig. 55

NOTICE

- Clean the Drive Shaft periodically with a cloth soaked in alcohol. Debris on Drive Shaft may cause incorrect pump operation.
- The pump is a wear item. If the irrigation volume decreases markedly, replace the pump.

10-3 Changing the Water Filter

If you use Tap Water, change the Water Filter once a year.

If you feel that the water volume has dropped, change the Water Filter before a year has elapsed.

- 1) Close the water valve of the dental unit.
- 2) Place the Control Unit on a cloth, etc. in case water leaks.
- 3) Use two spanners (5x8) and turn in the directions shown below in Fig. 56.
- 4) Remove the Water Filter Case and take out the Water Filter.
- 5) Place the new Water Filter with the O-ring on the Varios Combi Pro side and reassemble by following the disassembly steps in reverse (Fig. 57).

* Refer to "21 Spare Parts List" for details about the optional Water Filter.

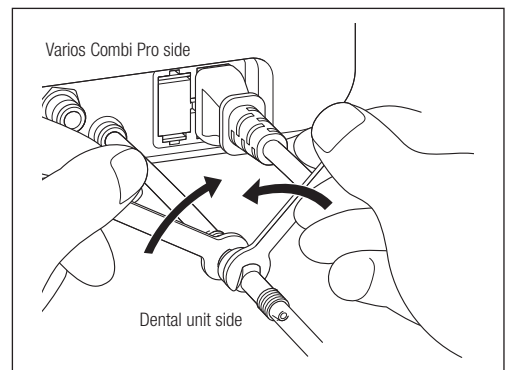
Fig. 56

CAUTION

- Make sure that the Water Filter is placed in the correct orientation. Otherwise the Filter will not work effectively.

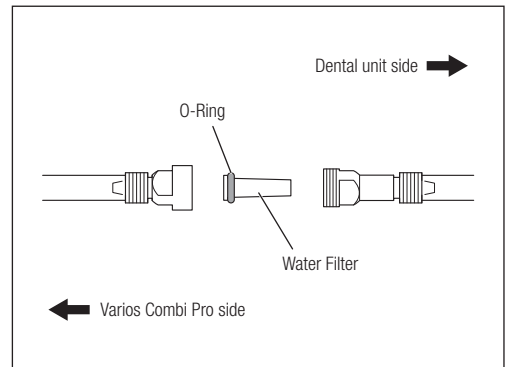
Fig. 57

10-4 Checking the Air Filter

When symptoms such as reduced Power Level or frequent clogging of powder are observed, release water by pressing the tip of the Air Filter at the bottom of the Control Unit (Fig. 58).

If sufficient Power Level cannot be obtained as a result of prolonged use, the VCP Element may have clogged. If this happens, contact your Authorized NSK Dealer.

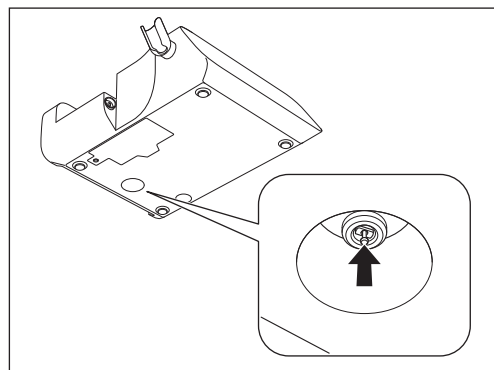
Fig. 58

10-5 Replacing the fuse

- 1) Turn OFF the Power Switch.
- 2) Unplug the AC power cord from the wall outlet.
- 3) Remove the Fuse Holder located at the back of the Control Unit while keeping the nails at both ends pushed inward (Fig. 59).
- 4) Replace the fuse with a new one.
- 5) Push the Fuse Holder back into the place.

* Refer to "21 Spare Parts List" for details about replacement Fuses.

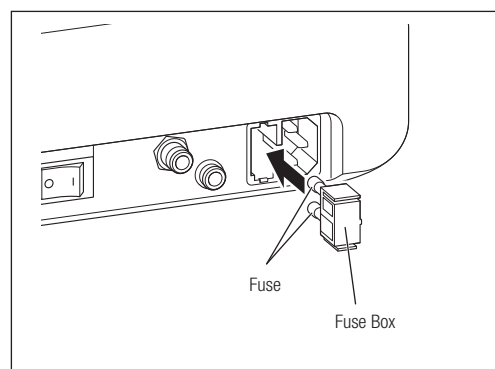
Fig. 59

- Only use NSK recommended fuses. Failure may occur if other fuses are used.
- If a fuse blows but you cannot establish the reason, contact your Authorized NSK Dealer.

NOTICE

- If the Fuse Box does not come out easily, use a hard-tipped implement to push the nails while pulling the Fuse Box out.

11 Other Functions

11-1 Initialized Program (Factory Setting)

The setting value for each of the modes are stored in this product. These values can be initialized to the factory settings.

- 1) Turn the POWER switch ON while depressing the Foot Control button B.
- 2) "Clr" is displayed on the LCD Panel. While "Clr" is displayed, release the Foot Control button B and then depress the button D to start initializing.
- 3) "Fin" is displayed on the LCD Panel when the initialization is complete. An alarm sound is heard and the product starts in the normal mode. (Refer to the table below for the initial settings.)

	Ultrasonic Mode	Powder Mode	
		Prophy Mode	Perio Mode
Ultrasonic Mode selection	Perio	-	
Power Level	1	1	1
Irrigation Mode	Bottle	- (fixed to tap water)	
Irrigation Volume	10	10	10

ULTRASONIC

11-2 Using the Tip Holder (for Ultrasonic Tip, option)

- Use the Tip Holder to store the Tips after removing them from the Handpiece.
- The Tip Holder is autoclavable. Up to 5 Tips can be sterilized at the same time. Rotate the Holder in the direction of the arrow so the Tips are set in the frame for sterilization (Fig. 60).

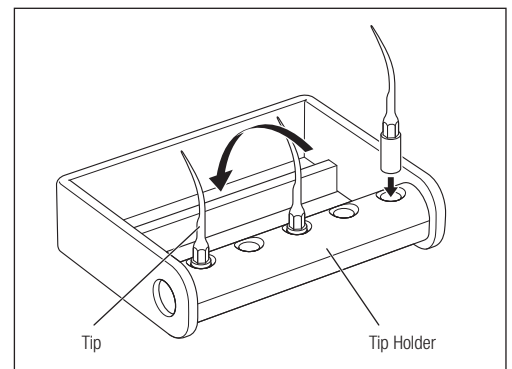
Fig. 60

11-3 Basic Water Level Setting

Basic Water Level can be adjusted depending on the supply water pressure in the usage environment. If the Irrigation Volume is too large or too small even when adjusted by the Irrigation Volume Keys, follow the procedure below to adjust the Basic Water Level.

- 1) Turn the POWER switch of the Control Unit to ON.
- 2) Touch the Operation Mode Select Key to select Powder Mode.
- 3) Touch and hold the Irrigation Mode Select Key for 2 seconds. Alarm sounds are heard and the LCD panel switches to the Basic Water Level adjustment mode.
- 4) Touch the Irrigation Volume Keys (▲/▼) to change the Basic Water Level ("L1" - "L5"). Adjust the Level as necessary.
- 5) Touch and hold the Irrigation Mode Select Key for 2 seconds to save the selected level to memory. An alarm sound is heard and the selected level is saved, and then the LCD Panel switches back to normal mode.

Fig. 61: Basic Water Level adjustment mode [Water Level 4]

NOTICE • Basic Water Level returns to "L4" when this product is initialized to the factory defaults. Set the Basic Water Level again after initializing.

Reference

Level	Water volume
L5	Large ↑ Small
L4 (Initial value)	
L3	
L2	
L1	

12 Periodical Maintenance Checks

Refer to the check sheet below and perform periodical maintenance checks every three months. Contact your Authorized NSK Dealer if any abnormalities are found.

Points to Check	Details
Ultrasonic Handpiece	Operate the Ultrasonic Handpiece to confirm there is no abnormal vibration, noise, heat, etc. Also check that the LED lights and water flows.
Chamber Cap	Check that there are no large/deep scratches.
Powder Spray	After filling the powder chamber, spray powder into a spittoon, etc., and confirm that powder is not leaking from connections other than the Nozzle Tip connection.
Water Spray	After filling the powder chamber, spray powder into a spittoon, etc., and confirm that water is not leaking from connections other than the Nozzle Tip connection.
Chamber	Check that there are no large/deep scratches.
Attaching/removing the Handpiece, Handpiece Hose, Irrigation Bottle and Chamber	Check that they can be attached and removed smoothly. If they are too loose or too tight, the O-rings may be damaged or deteriorated. Replace O-rings. (Refer to "10 Care and Maintenance.")

13 Error Code

"Error" and an Error Code are displayed on the LCD Panel of the Control Unit if this product stops operating due to malfunction, excessive pressure, disconnection, or incorrect operation. Depress the Foot Control button D to perform the error check again. If the problem is not detected, the error status is released and this product become operable. If the Error Code is still displayed, refer to the table below and take the necessary steps to resolve the problem.

Error Code	Description of Error	Check/Remedy
0	System Error	Turn on the Control Unit power again. Contact your Authorized NSK Dealer if the error does not disappear.
1	24V voltage abnormality	
2	12V voltage abnormality	
3	Pump current abnormality	Check the connection of the Irrigation Pump. If the problem still persists, contact your Authorized NSK Dealer.
4	Air pressure abnormality	Check the incoming air pressure. If the problem still persists, contact your Authorized NSK Dealer.
5	Heater voltage abnormality	Turn on the Control Unit power again. Contact your Authorized NSK Dealer if the error does not disappear.
6	Heater current abnormality	
7	Heater temperature abnormality	Check the water circuit and supply water to the Control Unit. Leave the Control Unit until it cools down before using it. Contact dealer if the error does not disappear.
8	Foot Control communication abnormality	Check the connection of the Foot Control. If the problem still persists, contact your Authorized NSK Dealer.
9	Ultrasonic Handpiece abnormality	Check the connection of the Handpiece. If the problem still persists, contact your Authorized NSK Dealer.
12	Touch sensor communication abnormality	Turn the POWER switch to OFF and then to ON. If the problem still persists, contact your Authorized NSK Dealer.
13	Error in self-check of the Ultrasonic Unit	
14	Ultrasonic Unit system error	

14 Troubleshooting

If you are experiencing problems, please check the following before contacting your Authorized NSK Dealer.

(Ultrasonic Mode and Powder Mode)

Problem	Cause	Remedy
Beeping sound	Power was turned on while the Foot Control was depressed.	Release the Foot Control and turn the POWER switch to ON.
	Abnormal heating of the Control Unit.	Stop operation and leave until Control Unit cools.
Touch panel does not react well or touched key is not recognized correctly.	Calibration is slightly misaligned due to static electricity, exogenous noise, etc.	Turn the POWER switch to OFF and then to ON.
	Key was not touched at the center.	Touch the center of the key.
<ul style="list-style-type: none"> Water does not stop The water level cannot be adjusted 	Damage or wear of the O-Ring.	Follow the procedure in "10-1-5 Chamber Base Connector" to replace the O-Ring.
	Foreign material has become clogged in the water route.	Perform Auto-Cleaning. If the symptom is still not resolved, contact your Authorized NSK Dealer.

(Ultrasonic Mode)

Problem	Cause	Remedy
Alarm sounds are heard, and the Ultrasonic Mode Display (P/E/G) and "ULTRASONIC" blink alternately when the Foot Control button D is depressed.	The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece.	Wipe the sensor clean.
No/Poor vibration.	The AC power cord plug is not connected to the wall socket or electricity is not supplied to the wall socket.	Check the connection.
	The Fuse is burned out.	Refer to "10-5 Replacing the fuse" and replace the fuse.
	The Tip is not tightened firmly.	Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out.
	Worn Tip.	Replace the Tip.
	The screw parts of the Handpiece and Tip are not clean.	Wipe them clean.
	The screw parts of the Handpiece are worn out as a result of a long time of use.	Contact your Authorized NSK Dealer.*
	Power has not been correctly adjusted for the Tip.	Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or on the Tip case label.
	The Foot Control is disconnected.	Connect the Foot Control Cord Plug correctly.
	Failure of vibration device in the Handpiece.	Contact your Authorized NSK Dealer.*
	Failure of internal components of the Foot Control.	Contact your Authorized NSK Dealer.*
The Tip is bent or broken.	The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece.	Wipe the sensor clean.
	Power has not been properly adjusted for the Tip.	Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label.
The Tip falls off.	The Tip is not tightened firmly.	Tighten the Tip until the Tip Wrench clicks.

Problem	Cause	Remedy
Noise from the Handpiece.	Power has not been properly adjusted for the Tip.	Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label.
	The Tip is not tightened firmly.	Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out.
	Failure of vibration in the Handpiece or the Control Unit.	Contact your Authorized NSK Dealer.*
The Handpiece is overheating.	Power has not been properly adjusted for the Tip.	Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label.
	The Tip is not tightened firmly.	Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out.
	The screw parts of the Handpiece and Tip are not clean.	Wipe them clean.
	The screw parts of the Handpiece are worn out as a result of a long time of use.	Contact your Authorized NSK Dealer.*
	The Irrigation Volume is insufficient or irrigation has stopped.	Irrigate at an adequate volume.
	Vibration failure in the Handpiece or the Control Unit.	Contact your Authorized NSK Dealer.*
No Irrigation supply and/or unstable Irrigation supply (when using the Bottle)	The Irrigation Tube is twisted.	Straighten the twisted Irrigation Tube.
	Time to replace the Irrigation Pump.	Replace with a new Irrigation Pump (Refer to 10-2 Changing the Irrigation Pump).
No/Poor water supply. (when using Tap Water)	Water is not supplied to the Control Unit.	Check the water circuit so that water is supplied to the Control Unit. Supply water pressure: 0.1 - 0.3MPa
	The Irrigation Volume is set to "0."	Adjust to the adequate Irrigation Volume using the Irrigation Volume setting.
	Water supply may be unstable when the Irrigation Volume is insufficient.	This is not a malfunction. Increase the Irrigation Volume in the Irrigation Volume setting.
	Water Filter is clogged with foreign substances.	Replace the water filter with a new one.
Water leakage.	The Irrigation Tube is not connected correctly.	Firmly insert the Irrigation Tube all the way into the Irrigation Connector.
	O-Ring at the Handpiece Cord is worn or damaged.	Replace with a new O-ring by following the procedures in "10-1-1 Handpiece Cord."
	The water circuitry in the Control Unit is damaged.	Contact your Authorized NSK Dealer.*
Handpiece LED does not light.	The Handpiece is not connected to the Handpiece Cord correctly.	Firmly insert the Handpiece all the way into the Handpiece Cord.
	Failure in the Control Unit.	Contact your Authorized NSK Dealer.*

Problem	Cause	Remedy
Alarms are heard, and "POWDER" and the tooth icon blink alternately when the Foot Control button D or B is depressed.	The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece.	Wipe the sensor clean.
	Chamber is not attached securely.	Attach the Chamber securely until the confirmation sound is heard.
Confirmation sound cannot be heard, or the icon that indicates Prophy Mode or Perio Mode is not displayed when the Chamber is attached.	Failure in the Control Unit (disconnection).	Contact your Authorized NSK Dealer.*
Air and powder are not supplied. Or air and powder are supplied but the pressure is not strong enough and cleaning power is also weak.	The inside of the Nozzle Tip is clogged with powder.	Replace the Nozzle Tip.
	The inside of the Handpiece is clogged with powder.	Clean the Nozzle base with the supplied Cleaning wire (small) and Nozzle end with the cleaning file. In addition, clean the inside of the Handpiece with the Cleaning wire (large). If powder and air do not flow after being cleaned as described above, perform Ultrasonic Cleaning with warm water. (Refer to "9-2-3 Ultrasonic Cleaning.")
	The inside of the Chamber is clogged with powder.	Remove powder from inside the Chamber by following the procedures in "9-4 Cleaning the Chamber and Hose."
	Powder has partially solidified with water, etc., in the air passage.	Remove powder from inside the passage by following the procedures in "9-2-1 Auto-Cleaning."
	Powder container has been left open exposing powder to the air.	Dispose of the powder and use new powder.
	Water is mixed in supply air.	Open the filter in the air supply circuit to drain the compressor, and check that water has not accumulated inside the air filter and compressor drain.
	The VCP Element is clogged.	Contact your Authorized NSK Dealer.*
Water leaks from between the Handpiece and Handpiece Hose.	The O-ring is scratched.	Replace the O-ring in the Handpiece joint section, apply a drop of the supplied turbine oil and rub it in with your finger. (Refer to "10-1-2 Handpiece Hose (for Prophy use and Perio use).")
Cannot remove the Chamber because the Lock Button cannot be pressed	The button is locked	Follow the procedure in "Releasing the Lock Button" in "9-4 Cleaning the Chamber and Hose" to release the Lock. If the symptom is not resolved, contact your Authorized NSK Dealer.*

* Cannot be fixed at clinics, etc.

15 Specifications

<Control Unit>

Model	Varios Combi Pro
Power source	AC100V - 240V 50/60Hz
Drive frequency	28 - 32kHz
Power input	78 - 98VA
Maximum output	11W
Supply air pressure	0.55 - 0.75MPa (5.5 - 7.5kgf/cm ²)
Supply water pressure	0.1 - 0.3MPa (1.0 - 3.0kgf/cm ²)
Bottle volume	400ml
Irrigation Volume (Ultrasonic Mode)	5.0 - 70.0mL/min (Irrigation Mode: Bottle/Tap water, Supply water pressure: 0.2MPa)
Dimension	W225 × D290 × H162mm (Including the Irrigation Bottle and Chamber)
Weight	2.7kg (Irrigation Bottle and Chamber not included)

<Foot Control>

Model	FC-79
Cord length	2.5m
Dimensions	W204 × D157 × H128mm (including Hanger)
Weight	415g (including Hanger)

<Ultrasonic Handpiece>

Model	VA2-LUX-HP
Cord length	1.8m (Connector not included)
Type of oscillation	Piezo type
Light	White LED
Dimensions	φ20 × 103mm (cord not included)
Weight	54g (cord not included)

<Powder Handpiece>

Model	PROPHY-HP/PERIO-HP
Hose length	1.8m (Connector not included)
Dimensions	φ16 × 107mm (cord not included)
Weight	33g (cord not included)

	Temperature	Humidity	Atmospheric pressure
Use environment*2	0 - 40°C	35 - 75%*1	—
Transportation and Storage environment	-10 - 50°C	10 - 85%	500 - 1,060hPa

*1 Without condensation

*2 The use of this product under conditions other than specified above may cause malfunction.

16 Classification of Equipment

- Type of protection against electric shock:
 - Class I equipment
- Degree of protection against electric shock:
 - Type B applied part: ⚡ (Applied parts: Ultrasonic Tip, Ultrasonic Handpiece, Nozzle Tip, Powder Handpiece)
- Method of sterilization or disinfection recommended by the manufacturer:
 - Autoclave sterilization
- Degree of protection against ingress of water as detailed in the current edition of IEC 60529:
 - Foot Control: IPX1 (Protected against vertically falling water drops)
- Degree of safety of application in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide:
 - Equipment NOT suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Mode of operation:
 - Continuous operation

17 Operation Principle

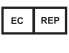
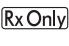
Ultrasonic Mode

A sinusoidal electrical signal is delivered by the generator at ultrasonic frequency ($f > 20\text{kHz}$). This signal is applied to the 'piezoelectric ceramic' located inside the transducer. Piezoelectric ceramic converts this signal into mechanical vibrations. These vibrations are at the same ultrasonic frequency as the electrical signal. The mechanical vibrations are propagated towards the distal end of the transducer. The "TIP" insert, which is attached at the distal end of the transducer, vibrates at ultrasonic frequencies and to achieve the purpose of the equipment.

Powder Mode

Sprays powder using air pressure to clean the tooth surface.

18 Symbol

	This product is Autoclavable up to Max. 135°C.		This product can be washed using Thermo Disinfector.
	Conforms to CE European Directive of "Medical device directive 93/42/EEC."		Manufacturer.
	Authorized representative in the European community.		Follow the waste of electric and electronic equipment (WEEE) Directive (2012/19/EU) for product and accessory disposal
	Consult operation instructions.		Type B applied part.
	Protected against dripping		For indoor use only.
	Marking on the outside of Equipment or Equipment parts that include RF transmitters or that apply RF electromagnetic energy for diagnosis or treatment		Temperature limitation.
	Do not reuse.		Use-by date
	Keep Dry.		Batch Code
	Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician		Serial number
	Sterile		Caution
	GS1 DataMatrix for Unique Device Identifier.		

19 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

20 Option Parts List

Model	Order Code	Remarks
MG-4H Multi Gauge	Z109400	
Tip Holder	Z221080	
Perio Set	Y1003042	Powder Handpiece (For Perio use) Chamber (For Perio use) Handpiece Hose (For Perio use) Perio-Mate powder Perio-Mate Nozzle Tip (40 pieces) Perio-Mate Nozzle Tip Remover

Model	Order Code	Remarks
Powder Handpiece (For Perio use)	T1077001	
Chamber (For Perio use)	U1135019	
Handpiece Hose (For Perio use)	T1053001	
Nozzle (For Perio use)	T1077012	
Perio-Mate powder	Y900938	2 bottles, each bottle contains 160g
Perio-Mate Nozzle Tip	Y1002741	40 pieces
Perio-Mate Nozzle Tip Remover	T1043751	
Water Pressure Reducing Valve	U1135093	

21 Spare Parts List

Model	Order Code	Remarks
FC-79	Z1256001	Foot Control
VCP Bottle Set	U1135017	Bottle and Cap (Irrigation Bottle, Irrigation Bottle Cap)
VCP Bottle Cap	U1135116	Cap only (Irrigation Bottle Cap)
VCP Bottle (Bottle only)	U1135099	Bottle only (Irrigation Bottle)
O-ring set (Irrigation Bottle)	Y1003544	One each of Thick and Thin O-ring
O-ring	Y1003546	1 for the Handpiece Cord
O-ring set (Chamber Base Connector)	Y1003435	
O-ring set (Handpiece Hose)	Y1003500	One each of Thick and Thin O-ring Common Parts for Prophy use and Perio use
O-ring set (Nozzle)	Y1003501	One each of Thick and Thin O-ring Common Parts for Prophy use and Perio use
O-ring	Y1003545	For Chamber Cap
Handpiece Cord	E351054	
Ultrasonic Handpiece	E351050	
Sterilization case	Z1035001	For Ultrasonic parts
Tip Wrench (CR-10)	Z221076	
Water Filter Set	U1135078	
Water Filter	U387042	
Spanner Wrench (5x8)	Y1001301	2
Irrigation Pump	U1135079	Connector ring excluded
FLASH pearl	Y900698	4 bottles, each bottle contains 300g
Prophy Set	Y1003771	Powder Handpiece (For Prophy use) Chamber (For Prophy use) Handpiece Hose (For Prophy use) FLASH pearl (5 sachets) Replacement O-ring
Chamber (For Prophy use)	U1135018	
Handpiece Hose (For Prophy use)	T1051001	
Powder Handpiece (For Prophy use)	T1074001	
Nozzle (For Prophy use)	T1074012	
Chamber Cap	U1135089	Cap only
Cleaning File	T798092	For the distal end and inside the Nozzle
Cleaning Wire (S)	T798095	For the root of the Nozzle
Cleaning Wire (L)	T798091	For parts other than those above
Cleaning Brush	T798093	
Unlocking tool	U1135256	
Ring Wrench	T1074601	
Turbine Oil	Z016112	
Air Hose	D0914406036	
Fuse	D1202220010	Rated value T2AH250V

22 Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

23 EMC Information (Electromagnetic Compatibility Information)

Guidance and manufacturer's declaration - Electromagnetic Emissions		
The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR11/EN55011	Group 1	The product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11/EN55011	Class B	The product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions EN/IEC61000-3-2	class A (except 120V)	
Voltage fluctuations/flicker emissions EN/IEC61000-3-3	Complies (except 120V)	

Guidance and manufacturer's declaration - Electromagnetic Immunity			
The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.			
Immunity test	EN/IEC60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) EN/IEC61000-4-2	±(2,4)6kV contact ±(2,4)8kV air	±(2,4)6kV contact ±(2,4)8kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst EN/IEC61000-4-4	±2kV for power supply lines ±1kV for input/output lines	±2kV for power supply lines ±1kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge EN/IEC61000-4-5	±1kV line(s) to line(s) ±2kV line(s) to earth	±1kV line(s) to line(s) ±2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines EN/IEC61000-4-11	<5% Ut (>95% dip in Ut) for 0.5 cycles 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 0.5 cycles <5% Ut (>95% dip in Ut) for 5 sec	<5% Ut (>95% dip in Ut) for 0.5 cycles 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 0.5 cycles <5% Ut (>95% dip in Ut) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the product requires continued operation during power mains interruptions, it is recommended that the product be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field EN/IEC61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: 'Ut' is the AC mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration - Electromagnetic Immunity			
The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.			
Immunity test	EN/IEC60601 test level	Compliance level	Electromagnetic environment - guidance

Conducted RF EN/IEC61000-4-6	3Vrms 150kHz to 80MHz	3Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the product, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz to 800MHz $d = 2.3 \sqrt{P}$ 800MHz to 2.5GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, and (d) is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters as determined by an electromagnetic site survey(a) should be less than the compliance level in each frequency range(b). Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF EN/IEC61000-4-3	3V/m 80MHz to 2.5GHz	3V/m	

NOTE 1: At 80MHz and 800MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level stated above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the product.

b: Over the frequency range 150kHz to 80MHz, the field strength should be less than 3V/m.

Cables and accessories	Maximum length	Complies with
Handpiece Cord	2.0m (Unshielded)	RF emissions, CISPR11 ClassB/Group1
Foot Control Cord	2.5m (Unshielded)	Harmonic emissions, (except 120V) EN/IEC61000-3-2
AC Power Cord	3.0m (Unshielded)	Voltage fluctuations/flicker emission, (except 120V) EN/IEC61000-3-3 Electrostatic discharge (ESD): EN/IEC61000-4-2 Electrical fast transient/burst: EN/IEC61000-4-4 Surge: EN/IEC61000-4-5 Voltage dips, short interruptions and voltage variations on power supply input lines: EN/IEC61000-4-11 Power frequency(50/60Hz) magnetic field: EN/IEC61000-4-8 Conducted RF: EN/IEC61000-4-6 Radiated RF: EN/IEC61000-4-3

Recommended separation distances between portable and mobile RF communications equipment and the product

The product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the product as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter	Separation distance according to frequency of transmitter m		
	150kHz to 80MHz $d = 1.2 \sqrt{P}$	80MHz to 800MHz $d = 1.2 \sqrt{P}$	800MHz to 2.5GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance “d” in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where “P” is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80MHz and 800MHz, the higher frequency range applies.

NOTE2 : These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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NSK Europe GmbH
NSK France SAS
NSK United Kingdom Ltd
NSK America Corp
NSK Oceania Pty Ltd